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## COMPRESSANA ULCER SYSTEM COMPRESSANA ULCER INTRA

Instructions for  
use and care

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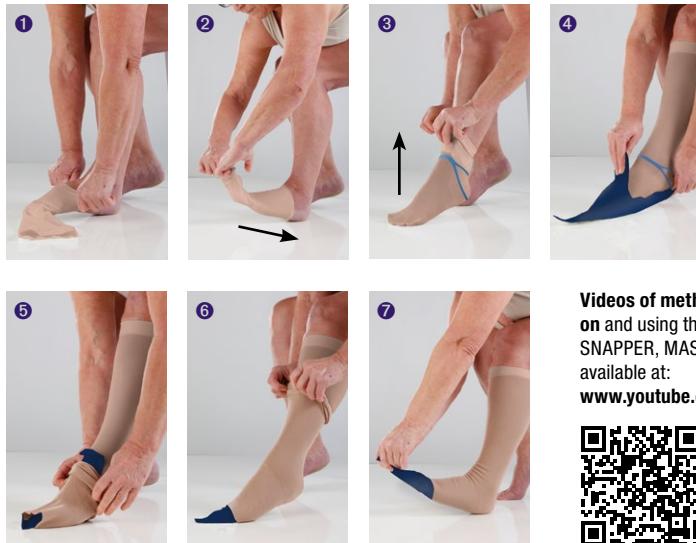
## Putting on

Videos of methods of putting on and using the COMPRESSANA SNAPPER, MASTER or EASY are available at:  
[www.youtube.com/compressana](http://www.youtube.com/compressana)

The COMPRESSANA ULCER SYSTEM is a medical compression stocking system for the treatment of ulcus cruris and consists of two compression stockings that can be combined:

**ULCER INTRA\***: The understocking with y-heel indication and closed foot tip serves to fixate the wound dressing or bandage. When pulling it on, ensure that the stocking has no wrinkles and that the bandage or wound dressing does not slip. The understocking can be worn on the affected leg day and night for continuous compression. Each package includes a second ULCER INTRA stocking for changing.

**ULCER FORTE**: The pull-over stocking with open toe serves to enhance the compression effect. The stocking is put on over the understocking and worn during the day.

## Tips for use

Fingernails and jewellery (e.g. sharp-edged rings) can damage the fine knitted fabric. Therefore it is best to use COMPRESSANA GRIP stocking gloves to protect your stockings when putting them on. With foot and corneal care, correctly cut toenails and the right footwear, you can avoid damaging your stockings.

## Putting on (image 1 – 7)

- ❶ To put on the understocking with y-heel indication and closed toe, slip it over your forearm and reach into the foot area from the inside with your fingers. By pulling it out, turn the stocking inside-out up to the heel. Stretch the foot opening with both hands and slide the foot section over the foot up to the instep.
- ❷ Gradually slip the understocking over the foot - simultaneously turning it back over to the right side. Make sure that the heel knit with the y mark is on the heel, as shown in image 12.
- ❸ Now slide the understocking evenly and without wrinkles upwards, without twisting or pulling at the edges, and flatten it against the leg with your flat hand at the same time. If you have pulled the understocking too far upwards, simply push it back to the ankle joint and then pull it upwards with less force.
- ❹ Now pull the included COMPRESSANA EASY gliding aid for open toe over the foot in such a way that the shorter flap lies over the instep/back of the foot and the longer flap lies against the sole.
- ❺ Then pull the overstocking over the COMPRESSANA EASY gliding aid until the knitted stocking is in the heel position. The end of the donning aid in the toe area must protrude from the stocking opening at the front.
- ❻ Now slide the entire overstocking upwards in sections towards the knee and simultaneously smoothen it against the leg with your flat hand. For optimal wearing comfort, position the end edge of the overstocking approx. 2 cm below the end edge of the understocking.
- ❼ Finally, remove the EASY gliding aid by pulling it out of the front of the foot part.

## Taking off (image 8 – 10)

Do not take off both stockings at the same time, but first fold down the overstocking with the open tip of the foot and strip it down the lower leg. Then carefully take it off, pulling it over the heel, instep and foot. Repeat these steps to take off the understocking.

## Please note:

Only wear your compression stockings according to a doctor's prescription. The stockings should only be provided by suitably trained personnel after instruction. Changes to the leg caused by illness or therapy make a new measurement and, if necessary, new production necessary.

Your COMPRESSANA compression stockings are largely resistant to oils, ointments, skin moisture and other environmental influences. Certain combinations may affect the durability of the knitted fabric. Therefore, a regular check in a pharmacy or in a medical specialist shop is recommended.

In the event of damage, please contact your pharmacy or specialist dealer. DO NOT repair the stockings yourself - this can impact the quality and medical effect.

For hygienic reasons, this product is intended for one person only.

\*ULCER INTRA understockings with closed toe and y-heel indication are also available separately (additional delivery set).

## Taking off

## Notes on ULCER INTRA understockings

- ❾ Please note that the toe seam of the understocking should be on the outside. This protects the sensitive skin in the toe area.
- ❿ Place the understocking according to the knitted y-heel mark and ensure that it does not wrinkle.

## Material composition

For precise information please see the label sewn in your compression stockings.

## Durability and storage options

Store at room temperature and in a dry place. Avoid heat and moisture. If you observe all directions in this brochure the expected life-time of your compression stockings will roughly amount to: 6 months of maximum wearing time. Usability/minimum durability period for series production:  see package label.

## Warranty

Since compression stockings are subject to wear due to use, wear and ageing, the warranty period is 6 months for use, but not longer than the expiry date indicated on the packaging. Tailor-made stockings should be used as soon as possible after manufacture and are not intended for storage. Thus they are not provided with a shelf life date.

Please keep the textile label and/or the package label.

The warranty also applies to the use of COMPRESSANA accessories.

## Risk class/medical product

Medical compression stockings are medical products and correspond to risk class 1.

In the event of product-related claims, such as damage to the knitted fabric or defects in the fit, please contact your medical dealer/healthcare professional directly. Only serious incidents that could lead to a significant deterioration in health or death must be reported to the manufacturer and the competent authority of the Member State. Serious incidents are defined in Article 2 (65) of Regulation (EU) 2017/745 (MDR).

## Quality Mark RAL

The RAL-GZ 387 quality mark for medical compression stockings guarantees safety through enhanced quality guidelines and test procedures. The production according to RAL is a prerequisite for the prescription and invoicing of compression stockings in Germany.

## Washing and care instructions

If possible, wash your compression stockings **daily**. Washing compression products not only serves hygiene, but also restores the shape and elasticity and thus the full medical effect. Wash your stockings separately the first time, as they may still stain.

ULCER  
Forte

40

In the **washing machine** with a mild detergent in a gentle cycle.

ULCER  
INTRA

60

We recommend using a COMPRESSANA laundry bag.

For **hand washing**, preferably use COMPRESSANA mild detergent in luke-warm water. Turn inside out before washing. Rinse out well, do no wring!

Air-dry, do **not** lay on a heater or in direct, strong sunlight! Dryers are to be used in the gentle drying cycle only.

Dry cleaning is **not** possible.

Chlorine bleach **cannot** be used.

Ironing is not necessary.  
Iron on the lowest level, do not iron hot

### Do not use fabric softener.

**Tips for quick drying:** Wrap stockings in a towel and squeeze them out well, e.g. by standing on them and pressing the moisture from the stockings into the towel using your body weight.

## Indications

- Ulcus treatment in venous leg ulcers instead of or in addition to compression bandages after oedema reduction
- Prevention of recurrent venous leg ulcers (recurrence prophylaxis)

## Contraindication

### Absolute contraindications

- Advanced peripheral arterial occlusive disease
- Decompensated heart failure
- Septic phlebitis
- Phlegmasia coerulea dolens

### Relative contraindications

- Pronounced weeping dermatoses
- Intolerance to compression stocking fabric
- Severe sensory disturbances of the limb
- Advanced peripheral neuropathy (e.g. Diabetes mellitus)
- Primary chronic arthritis

**If you do not observe these contraindications, COMPRESSANA assumes no liability.**

In the case of open wounds in the treatment area, the wound area must first be covered with an appropriate wound dressing

or bandage before the stocking system is donned. The skin in the treatment area must be checked regularly.

## Risks and side effects

When used properly, no side effects are known. If, however, negative changes should arise during the prescribed application, please contact your doctor or medical specialist immediately. If an intolerance to the ingredients of this product is known, please consult your doctor before use.

**The manufacturer is not liable for any damage or injury resulting from improper use, failure to follow the instructions or from the use of accessories or products not supplied by COMPRESSANA.**

## Disposal

There are no special disposal criteria. Compression products can be disposed with domestic refuse and the packaging material with paper recycling.

## Aids for putting on and off

COMPRESSANA put-on aids make it easier to put on and take off compression stockings. These are gentle on the skin and knit of the stockings. COMPRESSANA put-on aids are prescription aids.

### COMPRESSANA EASY donning aid leg

The simple and inexpensive put-on aid made of gliding cloth makes it possible to put on stockings without frictional resistance. EASY also makes it easier to dress wound covers or bandages on the foot or leg.

### COMPRESSANA SNAPPER donning aid

The donning aid makes it easier, faster and more comfortable to put on the stockings without effort, even with wet and lotioned legs.

### COMPRESSANA MASTER the station for putting on and taking off

In the case of reduced mobility, grasping ability and mobility, the special metal construction of COMPRESSANA MASTER allows effortless dressing and undressing in one - simply while sitting, without strenuously bending down and without any effort of the hands.

### COMPRESSANA GRIP stocking gloves

With COMPRESSANA GRIP stocking gloves you have a better grip to spread the knitted fabric evenly on the leg. It is also easy to correct the position of stretched or slipped stockings.

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## The way to achieve a successful therapy

- Due to the therapeutic qualities of compression stockings your doctor recommends them to alleviate vascular ailments, oedemas, lymphatic illnesses or scar treatment.
- Medical compression stockings are accurately coordinated with your anatomy. Their perfect fit and the skin friendly material makes it possible to wear Compressana products daily.
- COMPRESSANA stockings effectively counteract the development of vascular ailments and oedemas.
- The wide Compressana range enables you to select the suitable compression stocking for each opportunity. The stockings' compression characteristics and compression effect comply with the most recently acquired medical knowledge.

## Quality and fitting guarantee

6 months from the date of purchase we provide replacement free of charge for our support and compression products. This requires appropriate handling in compliance with our instructions for use and care, taking correct measurements and appropriate assessment. Warranty exclusion / limitation of reliability for usual signs of wear and inappropriate use.

For further information please see [www.compressana.de](http://www.compressana.de)

COMPRESSANA GmbH

Böhmerwaldstr. 3 · 93073 Neutraubling · Germany

Telefon: 0 94 01 / 92 26-0 · Fax: 0 94 01 / 92 26-20

E-Mail: [info@compressana.de](mailto:info@compressana.de) · [www.compressana.de](http://www.compressana.de)