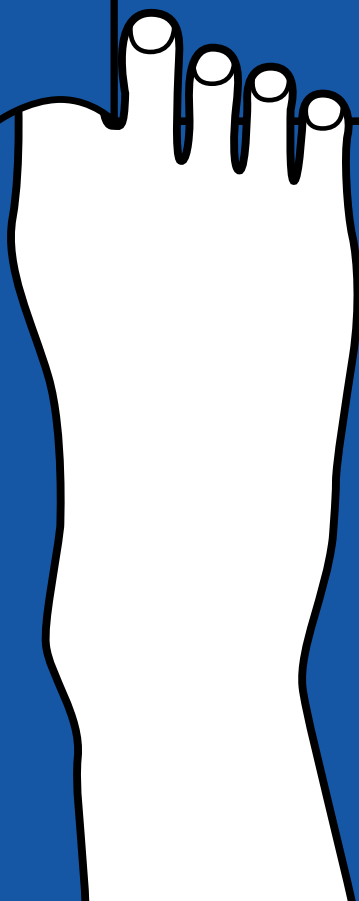
EASY

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EASY

INTENDED PURPOSE:

To support the veins and lymph vessels, compression stockings fit tightly like a second skin. A donning aid can make the application much easier. Owing to a special construction and slidability putting on and taking off according to the COMPRESSANA EASY is done quickly and without much strength required.

COMPRESSANA EASY is made of an especially silky fabric. Once slipped around the foot and leg, it allows you to pull on stockings even over moist or lotioned skin, wound covers or stockings worn underneath.

COMPRESSANA EASY is equally suited for open as well as closed toes.

In addition usage of special gloves with a grip on the palm is favourable, as recommended in the user manual of the hosiery manufacturers. These are available as accessories.

COMPRESSANA EASY protects the skin against injuries and the fabric against damage caused by mechanical influences, e.g. friction, fingernails, toenails or jewellery, especially in combination with COMPRESSANA GRIP special gloves. In this way the stockings keep their therapeutic effect.

COMPRESSANA EASY makes the start into the compression therapy easier and ensures that the stockings are donned correctly.

COMPRESSANA EASY is suitable for people who prefer an easier way of donning or people in special life circumstances, e.g. limited mobility.

Space-saving – ideal for travelling

Donning aids can be reimbursable when available on prescription.

Donning and doffing aids are recommended or prescribed by the doctor if the donning of the stockings without therapeutic appliances is too difficult. For example in cases of illnesses or disabilities with considerable dysfunctions of the of the mobility as well as for people with limited finger grip.

DETAILED INSTRUCTION!

Our partners in pharmacies or medical specialized trades will demonstrate how to use COMPRESSANA EASY and will be on hand with help and advice for the first instruction. In the beginning practice donning and doffing with COMPRESSANA EASY several times. After a short time you will gain routine in the handling. Please consider the following directions for donning and doffing as recommendations that offer the possibility to perfect the handling or to adapt it to your own style.

SPECIAL TIPS FOR HANDLING OF STOCKINGS WITH DONNING AIDS:

- Do not pull COMPRESSANA EASY out by the straps.
- Never pull the stockings at the top or at the top grip upwards.
- Upend the silicone beaded band before donning.
- In case of open wounds in the area of application, ensure that the wound is properly treated and covered.

For hygienic reasons, this product is intended for one person only.

DONNING


Variation 1:

COMPRESSANA EASY lies under the heel


Lay the donning aid COMPRESSANA EASY on the floor and place your foot on the cloth.

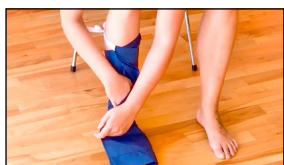Slip one or several toes into the provided opening in the hem.

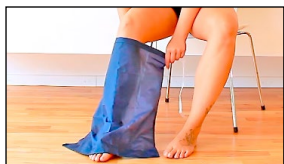
Loosely tie the cloth around your calf – yet not too high. Now wrap the cloth across your foot – first on the inside, then outside.


Variation 2:

COMPRESSANA EASY lies across the instep

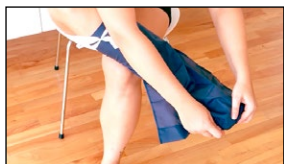Lay the donning aid COMPRESSANA EASY onto your foot and your shin.

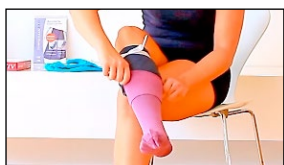
Loosely tie the cloth above your calf – yet not too high. Now slip the provided opening in the hem over one or several toes.


Next with both variations:

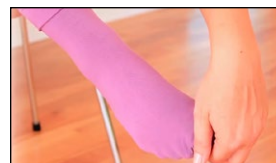
It helps to turn your stocking only halfway inside out. Slip the stocking over your foot and your donning aid.

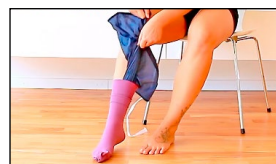
Undo the straps of the donning aid and lift the hem of the cloth over your toe.

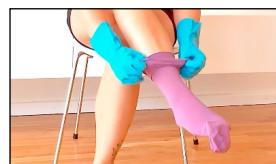Grab the cloth of the donning aid with both hands and pull it out of the stocking - bit-by-bit like a rope. **Do not pull on the straps.** Pulling it out can be easier by slipping the stocking back a bit.

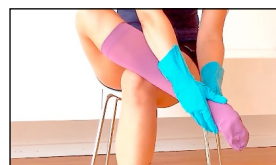Grab the stocking by the cuff and slip it up. Wear COMPRESSANA GRIP, TOP-GRIP or SUPER-GRIP gloves when doing so.


Evenly spread the knitted fabric with the gloves. Make sure that it lies flat and smooth, without any creases.

DOFFING

GRIP special gloves facilitate doffing of stockings and are available as accessories. Push the stocking down the leg by sliding the stocking step by step downwards with the help of the special surface texture of the gloves.

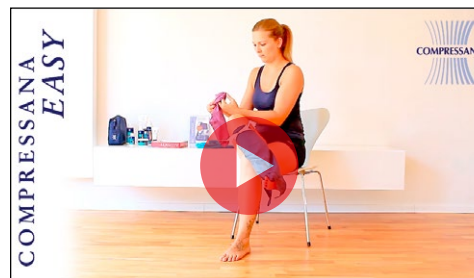Do never leave COMPRESSANA EASY on the floor.
Considerable danger of slipping!

Watch COMPRESSANA EASY in action:
www.youtube.de/compressana
or use the QR-Code with your mobile

MATERIAL COMPOSITION:

100% Polyester

WASH REFERENCES AND TEXTILE CARE SYMBOLS:

For simplest cleaning of COMPRESSANA EASY wipe with a damp rag.

Hand wash (approx..30° C).
Use mild washing detergent (COMPRESSANA CLEAN Art.No. 0121).

Air-dry or wipe dry with a cloth.

Do not bleach.

Do not dry-clean.

Do not iron. Do not expose to heat (e.g. radiator).

DURABILITY / DURATION OF USE / STORAGE:

Keep away from UV radiation. Store in a dry place and at room temperature. Avoid direct sunlight, heat and moisture. The application time depends on the intensity of use and the type of fabric or thickness of the stockings. Regular use results in a material-related fair wear and tear. Check COMPRESSANA EASY before use and application thoroughly for defects and proper functioning. In case of damage contact your specialist dealer or your pharmacy.

SAFETY NOTICE:

Please contact your specialist retailer or pharmacy in the event of damage. Do not repair the product yourself – this may affect its quality and effectiveness. Only use the donning aid when sitting. Carefully read the instructions for use and follow them. Keep the instructions in a safe place. The manufacturer does not accept any liability for damage/injuries resulting from improper use or misuse. Store COMPRESSANA EASY away from children.

INDICATIONS:

• Paralysis • Age-related reduction in strength • Osteoarthritis / rheumatism • Morbid obesity • Extensive stiffening of the spine/hip/knee • Degenerative disorders of the hands/in the area of the hand • Consequence of injuries/amputations

RELATIVE CONTRAINDICATIONS:

Absolute immobility.

SIDE EFFECTS:

There are no known side effects when this product is used correctly. However, if you notice any adverse changes during the prescribed use, please consult your doctor / pharmacy or medical retailer immediately. The manufacturer does not accept any liability for damage /injuries resulting from improper use or misuse. In the event of complaints related to this product, please contact the pharmacy / medical retailer directly. Only serious incidents that may lead to a major deterioration in the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR)

DISPOSAL:

There are no specific criteria for disposal. All components when worn-out or at the end of service life or lack of the ability to function can be disposed of with household waste, the packaging material with waste paper recycling.

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Art.Nr. C124/02 07/2023 © COMPRESSANA GmbH
COMPRESSANA GmbH