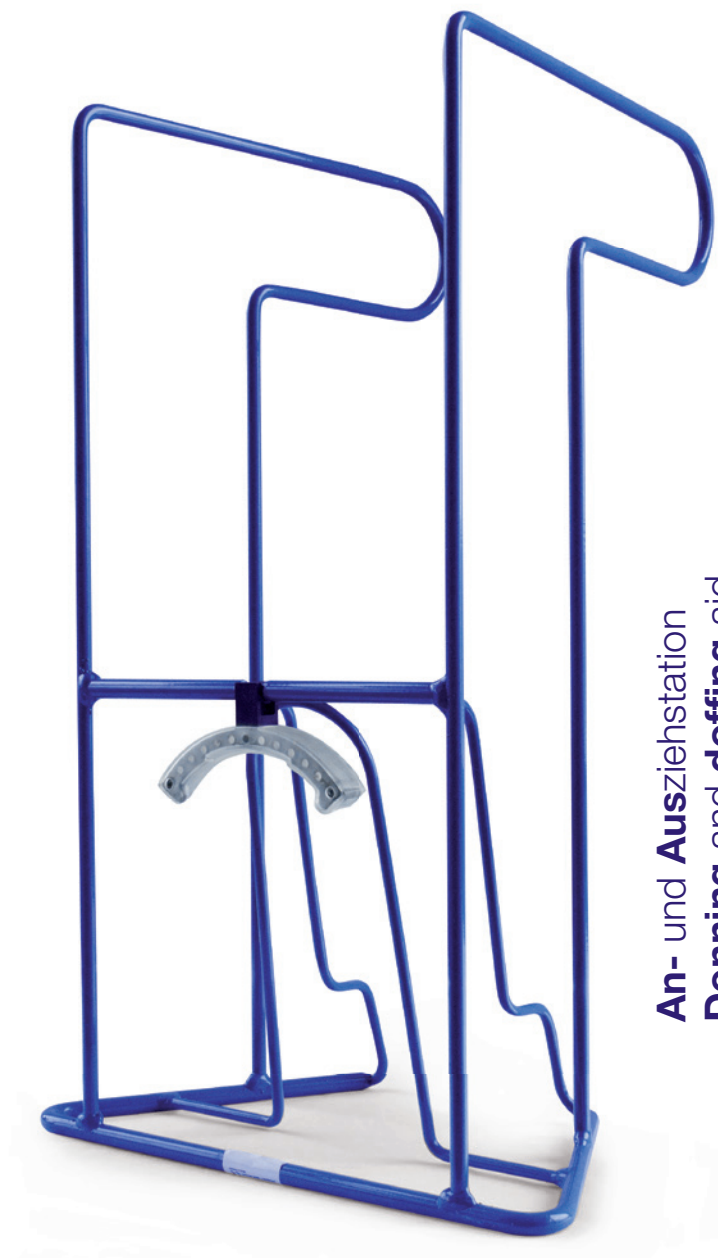
COMPRESSANA MASTER

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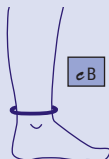
Intended purpose

To support the veins and lymph vessels, compression or support stockings fit tightly like a second skin. They are tightest at the bottom of the leg, at the ankle, in accordance with the human anatomy. To bring the stockings with the tightest fit over the foot and heel requires a special donning technique, strength and skill in the hands and good mobility, as the heel is furthest away from the hands. A **donning** and **doffing** aid can make the application considerably easier.

COMPRESSANA MASTER is a **donning** and **doffing** aid for compression or support stockings. The **donning** and **doffing** aid is particularly suitable for people with restricted mobility. It not only simplifies the process of putting on and taking off your medical compression stockings, but also gives you additional peace of mind: by using it, you avoid strenuous bending, and the risk of falling is reduced. The **donning** and **doffing** aid is also gentle both on your skin and the knitted fabric of your compression stocking. It stretches the stocking beforehand to make it easier to put it on. And the stocking clasp simplifies the process of taking it off.

COMPRESSANA MASTER The **donning** and **doffing** aid for for medical compression stockings

The COMPRESSANA MASTER makes it much easier to get in and out of the stockings:

- For easy donning, the stocking is pre-stretched by the donning station so that you can get into the foot / leg section and easily pull the stocking up to the knee.
- Taking off the stocking is facilitated by a holding element that secures the stocking for taking it off and a handle that „extends“ the arms.

Effortless **donning** and **doffing**

With a little practice, the COMPRESSANA MASTER allows you to put on and take off stockings

- **effortlessly and simply** • **sitting down** • **without strenuous bending** • **without effort**.

By pre-stretching the stocking material, friction on the skin is reduced and the skin is protected. In addition, the knitted fabric of the stockings is protected against damage.

The special suitability

COMPRESSANA MASTER is suitable for all people who prefer things to be easier or who have special circumstances or life situations, e.g. limited mobility etc. COMPRESSANA MASTER also makes it easier to start compression therapy and ensures that the stockings are put on correctly.

Donning and **doffing** aids are recommended or prescribed by the doctor for illnesses or handicaps with significant functional disorders of the finger, hand and arm joints, and with restricted freedom of movement, which make it impossible to put on stockings without aids.

COMPRESSANA MASTER is particularly suitable for circular knitted compression stockings or for support stockings.

Test without obligation!

If you are not sure whether the COMPRESSANA MASTER really makes things easier for you, you are welcome to test and try out the **donning** and **doffing** aid without obligation through our partners in pharmacies and specialist medical trade.

Thorough briefing!

Our partners in pharmacies or medical specialty stores will be happy to show you how to use the COMPRESSANA MASTER and are at your side for the first instruction.

Practice makes perfect!

Practice putting on and taking off with the COMPRESSANA MASTER several times in the beginning. With routine, you will be able to handle it after a short time, as if by magic. The following **donning** and **doffing** instructions are recommendations and offer the opportunity to perfect the handling or to adapt it to your own style.

Safety notice

The **donning** and **doffing** aid should be dispensed by properly trained staff only, at a medical retailer. Please contact your specialist retailer in the event of damage. Do not repair the product yourself – this may affect its quality and medical effectiveness. Only use the **donning** and **doffing** aid when sitting. Carefully read the instructions for use and follow them. Keep the instructions in a safe place. The manufacturer does not accept any liability for damage/injuries resulting from improper use or misuse. Store COMPRESSANA MASTER away from children. Only ever use COMPRESSANA MASTER if the frame and the clasp are in perfect condition..

Storage

Store dry under hygienic conditions. Avoid heat, direct sunlight and moisture.

Special instructions

- Never pull the stockings upwards by the top or adhesive band.
- Turn the adhesive tapes inside out before putting on the stockings.
- Wear stocking gloves to protect the fine knitted fabric from damage by fingernails, rings etc.
- Only use the COMPRESSANA MASTER when seated.
- In case of open wounds in the area of application, ensure that the wound is properly treated and covered.

Cleaning

Wipe the product with a damp cloth and conventional household cleaner.

Disinfection

If the device is used by different persons/users, e.g. in a practice, therapy or care, it can be disinfected by wiping with a surface disinfectant. Please follow the instructions on the disinfectant.

Indications

- Paralysis
- Age-related reduction in strength
- Osteoarthritis / rheumatism
- Morbid obesity
- Extensive stiffening of the spine/hip/knee
- Degenerative disorders of the hands/in the area of the hand
- Consequence of injuries/amputations

Relative contraindications

Absolute immobility

Liability for failure to comply with this contraindication cannot be accepted by COMPRESSANA GmbH.

Side effects

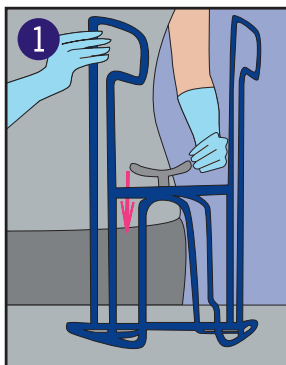
There are no known side effects when this product is used correctly. However, if you notice any adverse changes during the prescribed use, please consult your doctor / pharmacy or medical retailer immediately. The manufacturer does not accept any liability for damage / injuries resulting from improper use or misuse. In the event of complaints related to this product, please contact the pharmacy / medical retailer directly. Only serious incidents that may lead to a major deterioration in the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

Disposal


Disposal regulations may vary depending on your country. Therefore, please follow the national guidelines in the relevant country.


Using COMPRESSANA MASTER to **put on** your product

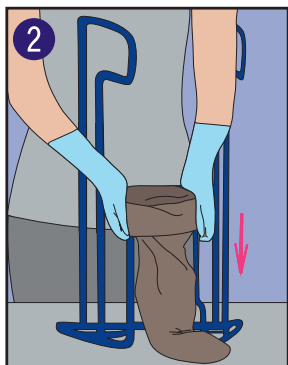
Fold the clasp back and down. Turn the COMPRESSANA MASTER until the clasp points on you.

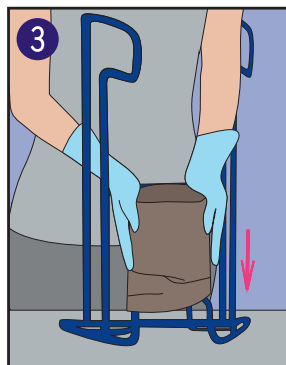
Using both hands, hold the stocking at the top and roll the edge of the stocking over the shaft. The toe of the stocking should be pointing away from the donning and doffing aid. COMPRESSANA special gloves facilitate handling.

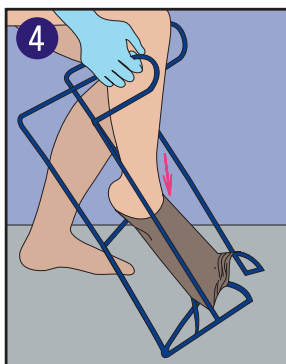
Now roll the stocking over the shaft until the seam of the heel is visible at the edge of the shaft.
Note for support stockings: Roll the stocking over the shaft until the tip of the foot is visible in the shaft.

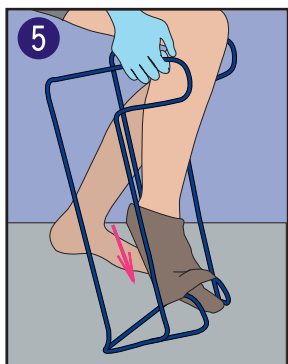
Now sit down and hold the donning and doffing aid handles at a slight angle. Make sure that your foot is dry. Starting with your toes, push your foot into the foot part of the stocking until your heel is no longer visible...


...slide your foot into the foot part until your toes are visible underneath the shaft. Now make sure the donning and doffing aid is almost vertical: raise your toes and apply pressure on the heel at the same time. This rolling motion will help your foot glide into the stocking with ease. Now firmly plant your foot on the floor. (Practise this motion sequence several times. It will make donning easier.)

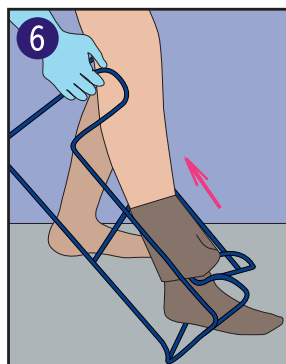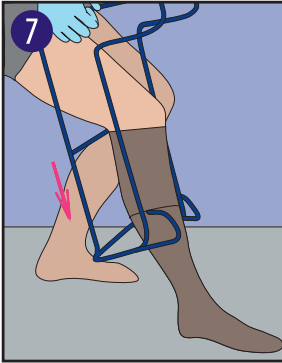
Using COMPRESSANA MASTER, pull the stocking up to knee-height.

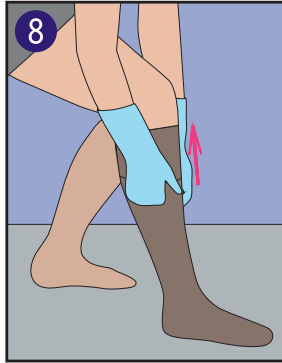Then carefully push the donning aid downwards and out of the stocking.

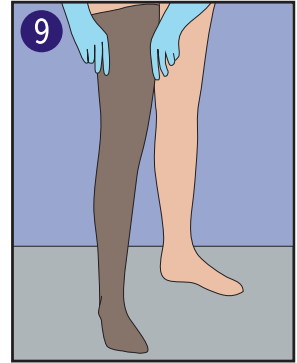Ease the stocking towards your thigh bit by bit with both hands.


Distribute the knitted fabric evenly up your leg, smoothing it out with gentle strokes as you go, and avoid creasing. Wear COMPRESSIONA special gloves to protect the fine knitted fabric from damage caused by finger nails, rings etc.


You can find a video showing the application of the COMPRESSIONA MASTER at www.compressana.de or you use the QR-Code with your mobile.

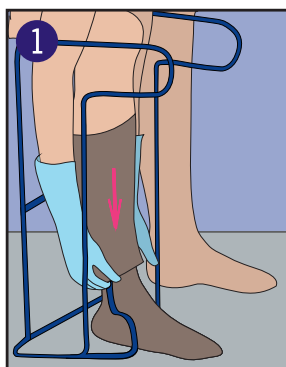
Using COMPRESSANA MASTER to **take off** your product

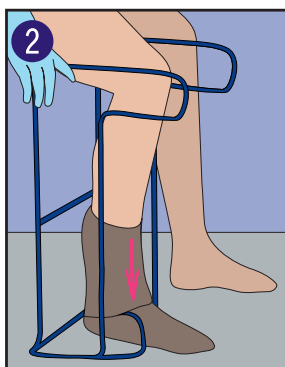
Hold the stocking with both hands and pull it down towards your knee. Now sit down and place your leg into the COMPRESSANA MASTER so that your calf is touching the upper curved edge of the shaft. Ease the top of the stocking down over the shaft until the top of the stocking is level with your heel.

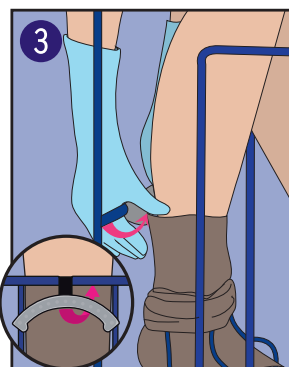
Push the entire stocking down as far as the shaft will permit.

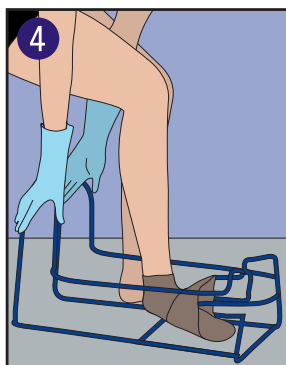
Using both hands, fold the clasp up towards the shaft until it clicks into place. The stocking is now secured.

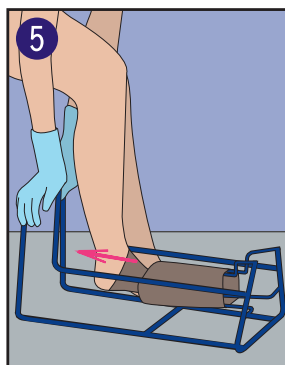
Place the COMPRESSANA MASTER on the floor. Hold both handles with your hands and, for additional stabilisation, put the other foot on one of the lower bars. Raise the foot with the stocking to shaft height. The heel should be above the clasp.

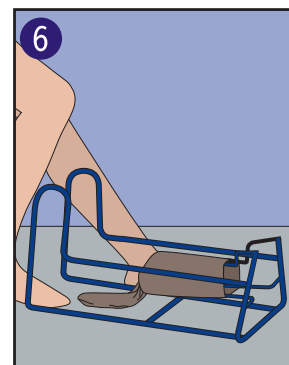Now pull your foot out of the stocking horizontally, parallel to the floor. Avoid pulling the foot up at an angle.

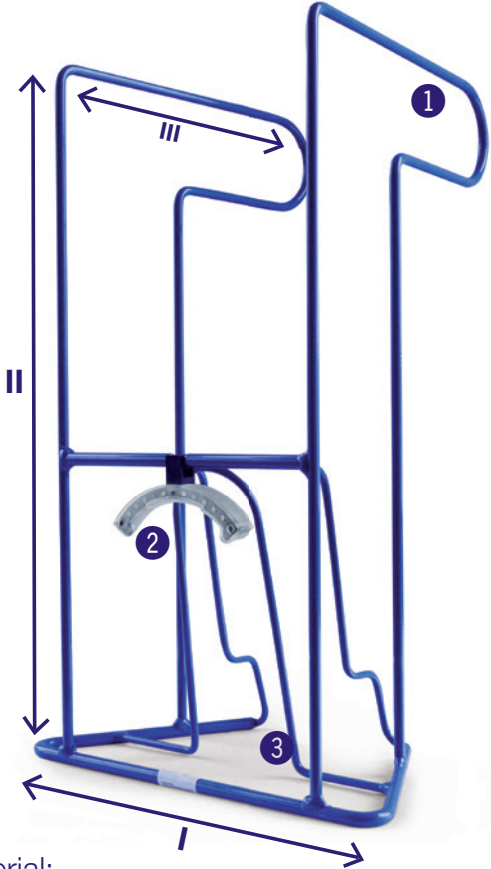Open the clasp by folding it downwards. Now you can remove the stocking easily from the COMPRESSANA MASTER.

Material:

Framework: lined metal • Clasp: PVC

Technical Data:

Measures:	I	32,0 x 18,0 cm
	II	52,0 cm
	III	28,5 cm

- ① Handles
- ② Clasp for take off
- ③ Shaft for pre-stretching

Tip for easier handling

The use of stocking gloves or powdering the feet and legs with talcum powder makes it much easier to put on stockings and tights.

COMPRESSANA GRIP

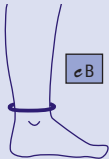
COMPRESSANA TOP-GRIP

COMPRESSANA SUPER-GRIP

B is measured above the ankle, at the narrowest point of the leg (shackle).

Product	Medical aids register-no.	PZN	Art.No.
Compressana MASTER Ankle circumference cB ≤ 26 cm	02.40.01.3028	09717627	0560
Compressana MASTER XL Ankle circumference cB > 26 cm	02.40.01.3028	07140218	0565
Oberflächendesinfektionstuch 10er-Set	—	16730746	0614

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