

FOR COMPRESSION
THERAPY





Make life more beautiful and colourful.
Compressana stockings are different - colourful,
individual and full of joie de vivre.

COLOURS COMPRESSION



The shades INDIGO and GRAPHIT become RISING
INDIGO and RISING GRAPHIT, respectively, by means
of elaborate dip dyeing, which become lighter and
lighter in the direction of the heart. In short: individual
and not reproducible - every stocking is unique.

AN OVERVIEW OF THE RANGE

PRODUCTS FOR THE COMPRESSION THERAPY – round knitted 4

COMPRESSANA LEGANZA – incomparably attractive	4
COMPRESSANA DIAMONDS – with diamond pattern	7
COMPRESSANA NOVA – uncompromisingly transparent	8
COMPRESSANA SENSITIV – incredibly smooth	10
COMPRESSANA LINEA – rib look	12
COMPRESSANA LINEA COTTON – sporty and modern	13
COMPRESSANA SPORT COTTON – sleek rib look	13
COMPRESSANA COTTON – consequently natural	14
COMPRESSANA FREE – easy and comfortable	16
COMPRESSANA FORTE – reassuringly effective	18
COMPRESSANA ULCER SYSTEM – simply practical	20

PRODUCTS FOR THE COMPRESSION THERAPY – flat knitted 22

COMPRESSANA FORTE pro	24
COMPRESSANA FORTE stabil	25
COMPRESSANA COTTON pro	26
DESIGNS AND AREAS OF APPLICATION of flat knitted products	27

COMPRESSANA SPECIAL MODELS 28

COMPRESSANA COMPRESSION PADS 29

COMPRESSANA SLEEVES 30

COMPRESSANA SEGMENTS + TOE PART 31

COMPRESSANA SUPPLIES 32

INTRA BASE LAYER STOCKINGS	32
MASTER Donning and doffing aid	34
SNAPPER Donning Aid	35
EASY Donning Aid	36
SPECIAL GLOVES	37
FIXATION	38
LYMPHATIC CUSHION	39
SKIN CARE	40
ACCESSORIES FOR CARE, WASHING AND STORAGE	43
INSHOE-SOCKS AND STEPS	45
GENERAL OVERVIEW	46

COMPRESSANA LEGANZA

incomparably attractive



TENDER TO LEGS
ATTRACTIVE IN APPEARANCE

- **exciting, highly sensitive**
- **extremely fine and silky shining**
- **even, smooth surface structure**

By the use of high tech yarns and due to an optimized manufacturing and finishing process an unequalled homogeneous and fine mesh surface is obtained. COMPRESSANA Leganza convinces with a favourable smooth surface!

The very fine texture causes a surprisingly pleasant feeling and a marvellous softness on the skin. COMPRESSANA Leganza is smooth and soft.

Compressana Leganza offers due to its innovative material properties a superior appearance. Compression yarns in its natural state facilitate a discreet transparency and accentuate thereby the natural skin tone.

Particularly to the anatomy adapted shaping and outstanding stretching characteristics ensure optimal wearing comfort in every condition of life. Furthermore COMPRESSANA Leganza convinces with a favourable accentuation of the legs and reveals a feminine silhouette.

CCL 1 light compression
2,4 - 2,8 kPa (18 - 21 mmHg)
Art.No. 160 (closed toe)
Art.No. 161 (open toe)






CCL 2 moderate compression
3,1 - 4,3 kPa (23 - 32 mmHg)
Art.No. 260 (closed toe)
Art.No. 261 (open toe)



closed toe



open toe

Style	
 A-d A-d short A-d extra short	Below-knee stockings
 A-d/H A-d/H short A-d/H extra short	Below-knee stockings <ul style="list-style-type: none"> • Silicone band (3,5 cm) • Softcontact band (3,5 cm)
 A-g/H A-g/H short A-g/H extra short	Thigh-high stockings <ul style="list-style-type: none"> • Silicone band (dots) • Lace band • Silicone dots w/ pattern • Softcontact band
 A-t A-t short A-t extra short	Tights asymmetric panties, Body extends standard size approx. 80-120 cm
 A-t/G A-t/G short A-t/G extra short	Maternity tights



NEW: Softcontact band with microdots for best grip with low pressure load

Colours			
white	sandel	silk	perle
choco	graphit	indigo	black

COLOURACCENTS &
COLOURGRADIENTS

COLOURS Compression valid until 01/2022


enzia	viola	malva
-------	-------	-------

Gradient Colours valid until 01/2022

rising indigo	rising graphit
---------------	----------------

Extravagant colour gradient decreasing towards the heart for thigh-high stockings and tights

COMPRESSANA LEGANZA & DIAMONDS Perfect fit - Superior wearing properties

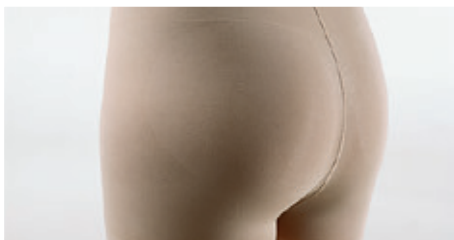

Panty

- anatomically shaped panty
- different elasticity and stretch zones
- thin meshes and non-bulky fabric
- flat waistband, convenient and comfortable




Thigh zone

- flexible, high elastic thigh zone with stretch reserves
- feminine silicone top bands in Italian design, sewn without overlapping




Foot

- completely reinforced sole without restriction of the longitudinal elasticity
- spacious, non-constricting toe with soft stretch
- soft and particular fine seams at the toe

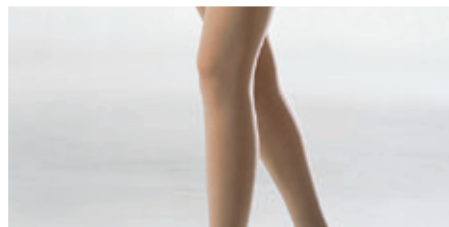

Knee and calf zone

- soft, elastic, non-constricting in the hollow of the knee
- below-knee stockings with seamless soft-cuff

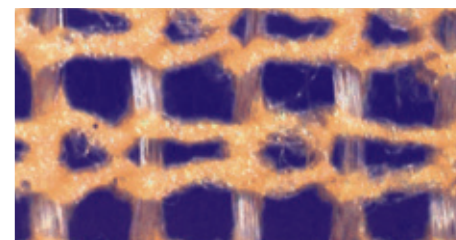

Knitting

- natural yarns for feminine gloss and delicate appearance
- softness, transparency and unusual homogeneous and fine mesh surface by innovative yarn and knitting technology
- knitted on most modern knitting machines

Convincing in application and handling

- streak-free and without colour transitions
- gently coloured and elaborately refined
- climate, temperature and moisture regulating characteristics
- easy-care, suitable for machine wash and tumble dryer

COMPRESSANA DIAMONDS

with diamond pattern

DIAMONDS ARE A GIRLS BEST FRIEND

COMPRESSANA DIAMONDS with a knitted diamond pattern and fine silky mesh show off your legs confidence. The modern way of compression therapy: Invisible effect – visibly attractive!
COMPRESSANA DIAMONDS is available in standard sizes.

CCL 1 light compression
2,4 - 2,8 kPa (18 - 21 mmHg)
Art.No. 168





CCL 2 moderate compression
3,1 - 4,3 kPa (23 - 32 mmHg)
Art.No. 268



closed toe

Colours



Style	
 A-d A-d short A-d extra short	Below-knee stockings <ul style="list-style-type: none"> • Silicone band (3,5cm) • Softcontact band (3,5cm)
 A-g/H A-g/H short A-g/H extra short	Thigh-high stockings <ul style="list-style-type: none"> • Softcontact band
 A-t A-t short A-t extra short	Tights asymmetric panties, Body extends standard size approx. 80-120 cm
 A-t/G A-t/G short A-t/G extra short	Maternity Tights

COLOURACCENTS

COLOURS Compression valid until 01/2022



COMPRESSANA
NOVA
uncompromisingly transparent



TRANSPARENCY
IN COMPLETION

- particularly transparent
- airy and fine knitting
- ideal in summer

Inconspicuous appearance, air permeability, transparency and perfect fit makes COMPRESSION NOVA unique among compression stockings. Our customers also think so:

I would like to say a big praise for NOVA at this point. So far - and I have tried quite a few models over the years - these are the first stockings that are really comfortable to wear, that last all day without me having to pull them up. I can wear them even in warm temperatures without feeling uncomfortable. They are also much less susceptible to running. Thank you so much for bringing this model to the market, you have done a great service to many people and made compression therapy HIGHLY more comfortable!

As an addendum, I would like to say that NOVA stockings are also the best in terms of fashion. I'm 39 and I certainly don't feel like walking around in shiny or totally opaque granny stockings. I wear NOVA normally with skirts and it is not at all noticeable that they are compression stockings.

Many greetings M.B.

CCL 1 light compression
2,4 - 2,8 kPa (18 - 21 mmHg)
Art.No. 180 (closed toe)
Art.No. 181 (open toe)





CCL 2 moderate compression
3,1 - 4,3 kPa (23 - 32 mmHg)
Art.No. 280 (closed toe)
Art.No. 281 (open toe)



closed toe



open toe


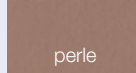
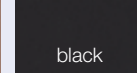
Style	
 A-d A-d short A-d extra short	Below-knee stockings
 A-d/H A-d/H short A-d/H extra short	Below-knee stockings <ul style="list-style-type: none"> • Silicone band (3,5cm)
 A-g/H A-g/H short A-g/H extra short	Thigh-high stockings <ul style="list-style-type: none"> • Silicone band (dots) • Lace band • Silicone dots w/ pattern
 A-t A-t short A-t extra short	Tights asymmetric panties, Body extends standard size approx. 80-120 cm



attractive lace-band



anatomical perfect toe with flat seam

Colours		
		
silk	perle	black

for
LADIES &
GENTLEMEN



COMPRESSANA SENSITIV

incredibly smooth



VELVET AND
UNBELIEVABLY GENTLE

- unequalled soft and gentle
- especially comfortable to wear
- temperature and moisture regulating, breathable

COMPRESSANA SENSITIV coats the leg with a fine woven and soft knitting. COMPRESSANA SENSITIV dons easily and grants maximum wearing comfort. The compression spreads itself gently and evenly. Annoying pinching and feeling of pressure does not exist any longer. You immediately forget wearing a compression stocking. Soft touch and smooth on the skin. The high-quality, double covered yarns flatter the skin and offer best skin compatibility, skin protection and long durability.

The stockings are breathable and moisture regulating. The even and concealing fabric offers ladies as well as gentlemen a visually attractive appearance.

CCL 1 light compression
2,4 - 2,8 kPa (18 - 21 mmHg)
Art.No. 195 (closed toe)
Art.No. 190 (open toe)

CCL 2 moderate compression
3,1 - 4,3 kPa (23 - 32 mmHg)
Art.No. 295 (closed toe)
Art.No. 290 (open toe)



closed toe



open toe

Colours

white

sandel

silk







perle

choco

graphit

indigo

black

Style	
	A-d A-d short A-d extra short Below-knee stockings
	A-d/H A-d/H short A-d/H extra short Below-knee stockings <ul style="list-style-type: none"> • Silicone band (3,5 cm) • Softcontact band (3,5 cm)
	A-g A-g short A-g extra short Thigh-high stockings
	A-g/H A-g/H short A-g/H extra short Thigh-high stockings <ul style="list-style-type: none"> • Silicone band (dots) • Lace band • Silicone dots w/ pattern • Softcontact band
	A-g/T A-g/T short Thigh-high stockings with waist attachments
	A-t A-t short A-t extra short Tights Highly elastic panty with adjustable waistband Body extend approx. 80-140 cm



COLOURACCENTS

COLOURS Compression valid until 01/2022

enzia

viola

malva

COMPRESSANA LINEA

rib look

Invisible effect – visibly attractive. COMPRESSANA LINEA ensnares and styles in a particularly noble way and draws attention to the legs. No one will know that you wear compression stockings!! COMPRESSANA LINEA combines the unrivalled soft and gentle wearing comfort of SENSITIV with an airy and attractive rib structure. Below-knee-stockings – modern and sportively elegant.

CCL 1 light compression
2,4 - 2,8 kPa (18 - 21 mmHg)
Art.No. 196

CCL 2 moderate compression
3,1 - 4,3 kPa (23 - 32 mmHg)
Art.No. 296



closed toe

Style



A-d
A-d short
A-d extra short

Below-knee stockings
with rib structure



A-d/H
A-d/H short
A-d/H extra short

Below-knee stockings
with rib structure,
Silicone band (3,5 cm)



also gentlemen love it

Colours

silk

choco

graphit

indigo

black

COMPRESSANA LINEA COTTON

sporty and modern

COMPRESSANA LINEA COTTON with rib structure provides below-knee stockings with a modern and sporty appearance. Inconspicuously, so that nobody can recognize by the mesh that you are wearing compression stockings.

Also available with grip top for double secure grip. The silicone beaded band is slip-resistant and suitable for highest demands. COMPRESSANA LINEA COTTON is ideal for moving, active legs due to the unique and skin-friendly style with high proportion of cotton.

COMPRESSANA LINEA COTTON is advantageous to other compression stockings. The graduated compression and efficient penetration increase performance and endurance.

CCL 1 light compression
2,4 - 2,8 kPa (18 - 21 mmHg)
Art.No. 156



CCL 2 moderate compression
3,1 - 4,3 kPa (23 - 32 mmHg)
Art.No. 256



closed toe

Style		
	A-d A-d short A-d extra short	Below-knee stockings with rib structure
	A-d/H A-d/H short A-d/H extra short	Below-knee stockings with rib structure • Silicone band (3,5 cm) • Softcontact band (3,5 cm)

Colours

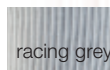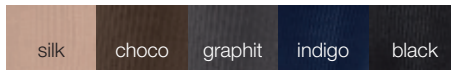

COMPRESSANA SPORT COTTON

Performance and endurance

The COMPRESSANA LINEA COTTON Racing Style offers high comfort and outstanding features for ambitious athletes and sportsmen.

- better blood circulation and oxygen supply
- activation and stabilisation of the musculature
- faster recovery
- reducing of the risk of injury

A man and a woman are running through a field of tall grass. The man is carrying the woman piggyback. They are both smiling and looking towards the right. The background is a blurred field of tall grass under bright sunlight.

COMPRESSANA COTTON

consequently natural



THE NATURAL SOLUTION FOR LEG PROBLEMS

- **soft, smooth cotton inside**
- **air-permeable and very skin friendly**
- **hardwearing by abrasion-resistant exterior**

COTTON is the skin-friendly option for compression stockings. The high cotton proportion provides a particularly pleasant and natural wearing comfort. COMPRESSANA COTTON is suitable for persons who suffer from dry and itching skin or who prefer natural materials.





The speciality of COMPRESSANA COTTON is the processing. Due to a special technology the cotton is located only on the inside of the stocking. This results in an almost 100% coverage of the legs with natural fibres, while the wearresistant synthetic fibres protect the exterior of the stockings against fast abrasion.





COMPRESSANA COTTON is the ideal compression stocking for all, who would like to have a natural feeling on the skin.

CCL 1 light compression
2,4 - 2,8 kPa (18 - 21 mmHg)
Art.No. 150

CCL 2 moderate compression
3,1 - 4,3 kPa (23 - 32 mmHg)
Art.No. 250

Style

	A-d A-d short A-d extra short	Below-knee stockings
	A-d/H A-d/H short A-d/H extra short	Below-knee stockings <ul style="list-style-type: none">• Silicone band (3,5 cm)• Softcontact band (3,5 cm)
	A-f A-f/H	Half-thigh-high stockings <ul style="list-style-type: none">• Silicone band (dots)• Lace band• Silicone dots w/ pattern• Softcontact band
	A-g A-g short A-g extra short	Thigh-high stockings

	A-g/H A-g/H short A-g/H extra short	Thigh-high stockings <ul style="list-style-type: none">• Silicone band (dots)• Lace band• Silicone dots w/ pattern• Softcontact band
	A-g/T A-g/T short	Thigh-high stockings with waist attachments
	A-t/L A-t/R	One-legged pantyhose with leg extension left or right
	A-t A-t short A-t extra short	Tights with adjustable waistband. Body extends standard size approx. 130 cm



Silicone band




closed toe




open toe

Colours

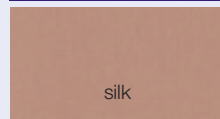

silk

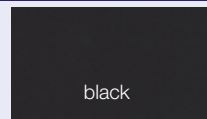

black

COMPRESSANA FREE

easy & comfortable



FOR MORE FREEDOM IN THERAPY

- unsurpassed easy donning and doffing
- more mobility
- pleasant and comfortable
- better hold, less slipping

Innovative knitting technology with integrated functional zones, sliding yarn technology and controlled elasticity for a new feeling of compression. Due to its diverse properties and benefits, COMPRESSANA FREE is suitable for a very wide range of indications for men and women of all ages and all constitutions and offers noticeable therapy relief even for beginners or people with physical disabilities/restrictions.

Easy to put on and take off, more freedom of movement, a suitable look for everyday use as well as soft, skin-friendly and breathable yarns offer uncomplicated handling and facilitate daily use and promote mobility.

COMPRESSANA FREE enables noticeably easier compression therapy with maximum comfort - when putting it on and wearing it.

CCL 1 light compression
2,4 - 2,8 kPa (18 - 21 mmHg)
Art.No. 170

CCL 2 moderate compression
3,1 - 4,3 kPa (23 - 32 mmHg)
Art.No. 270



closed soft toe



open toe

Colours

sandel*

silk

graphit*

indigo*


black

* expected from summer 2021

Style		
	A-d A-d short A-d extra short	Below-knee stockings
	A-d/H A-d/H short A-d/H extra short	Below-knee stockings • Silicone band (3,5 cm) • Softcontact band (3,5 cm))
	A-g A-g short A-g extra short	Thigh-high stockings*
	A-g/H A-g/H short A-g/H extra short	Thigh-high stockings* • Silicone band (dots) • Lace band • Silicone dots w/ pattern • Softcontact band

 **innovative functional zones**

- for an easy & pleasant everyday life
- for a better wearing feeling & noticeable therapy efficiency

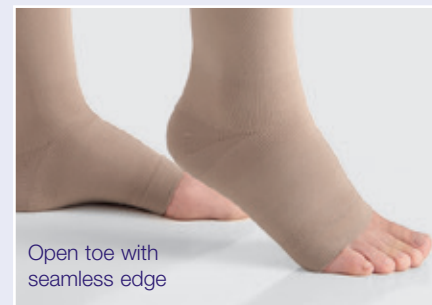

COMPRESSANA FORTE

reassuringly effective



STRONG AND STURDY IN
EVERY CONDITION OF LIFE

- durable and long lasting
- easy donning
- soft and smooth
- outstanding elasticity
- extraordinary in the knee and thigh-part

The stocking for everybody for whom function counts most. With Compressana FORTE you uncompromisingly decide for a stocking with exceptional effectiveness with which you walk through thick and thin. Strong, covered yarns constitute firm and nevertheless pleasant compression stockings. With strong working pressure COMPRESSANA FORTE is also suitable for men. The widemeshed and soft knitting benefits the air circulation, transports moisture and stimulates the blood circulation.





A further advantage: donning FORTE is particularly easy as the sturdy, wear-resistant material can be handled easily.





CCL 1 light compression
2,4 - 2,8 kPa (18 - 21 mmHg)
Art.No. 130

CCL 2 moderate compression
3,1 - 4,3 kPa (23 - 32 mmHg)
Art.No. 230

CCL 3 firm compression
4,5 - 6,1 kPa (34 - 46 mmHg)
Art.No. 330

Style

	A-d A-d short A-d extra short	Below-knee stockings
	A-d/H A-d/H short A-d/H extra short	Below-knee stockings <ul style="list-style-type: none">• Silicone band (3,5 cm)• Softcontact band (3,5 cm)
	A-f A-f/H	Half-thigh-high stockings <ul style="list-style-type: none">• Silicone band (dots)• Lace band• Silicone dots w/ pattern• Softcontact band
	A-g A-g short A-g extra short	Thigh-high stockings

	A-g/H A-g/H short A-g/H extra short	Thigh-high stockings <ul style="list-style-type: none">• Silicone band (dots)• Lace band• Silicone dots w/ pattern• Softcontact band
	A-g/T A-g/T short	Thigh-high stockings with waist attachments
	A-t/L A-t/R	One-legged pantyhose with leg extension left or right
	A-t A-t short A-t extra short	Tights with adjustable waistband. Body extends standard size approx. 130 cm



open toe



closed toe



open toe


Colours

sandel	silk	graphit	indigo	black
--------	------	---------	--------	-------

COMPRESSANA ULCER SYSTEM

simply practical

For more information on the
ULCER SYSTEM, please see
our customer information
(Art.No.: C199).

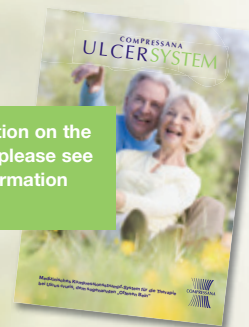

WITH COMPRESSION TO SUCCESS

**Medical 2-component compression
stocking system for the treatment of
venous leg ulcer.**

In the case of „open leg“, after the decongestion phase of the oedema, the treatment should be switched to two-layer ulcer compression systems for long-term therapy, if possible. The COMPRESSANA ULCER SYSTEM consists of an overstocking and two understockings that can be changed. When worn in combination, the compression pressure of ULCER FORTE and ULCER INTRA add up to a resting pressure of compression class 3. The stocking components are optimally matched to each other and easy to use: The understocking with light compression is used to fix wound dressings and pads. It can be worn on the affected leg day and night for permanent compression. The medium compression overstocking is applied over the understocking during the day to increase compression.

Compression in combination with movement ensures treatment success: COMPRESSANA ULCER SYSTEM offers non-slip wearing comfort, full freedom of movement and mobility. Normal sports clothing and shoes can be worn. Even after the venous leg ulcer has healed, medical compression stockings with high working pressure and depth effect remain indispensable. The COMPRESSANA ULCER SYSTEM can also be used for relapse prophylaxis.

CCL 3 firm compression
4,5 - 6,1 kPa (34 - 46 mmHg)
Art.No. 333

ULCER SYSTEM

1 Overstocking ULCER FORTE CCL 2 with open toe + 2 Underliner ULCER INTRA CCL 1 with closed toe

Colours

silk

CCL 1 light compression
2,4 - 2,8 kPa (18 - 21 mmHg)
Art.No. 183

ULCER INTRA (Additional delivery set)

2 Underliner ULCER INTRA CCL 1 with closed toe

(Made-to-measure: 1 Underliner)

Style



A-d
A-d short

Below-knee stockings



Overstocking with open toe and
Underliner with closed toe



Underliner: The heel/instep marking helps to
ensure exact positioning



HIGHER WEARING COMFORT
AND PERFECT FIT
STITCH BY STITCH

THERAPEUTIC FLAT KNITTED COMPRESSION GARMENTS

The compression therapy focuses on venous diseases (phlebology). For this treatment round knitted compression stockings in the prevailing compression classes 1 and 2 (ccl1 and ccl2) are used. Here primarily ready to wear sizes for anatomically normal legs are applied.

Flat knitted compression products up to ccl IV are almost only custom-made.

Field of application:

- in lymphology for medical care of lymphoedema and lipoedema, and for mixed forms
- such as lymphoedema combined with venous disease
- scar treatment, especially during therapy of deep burns and scalds
- treatment of anatomically complex and problematic body proportions that cannot be treated with circular-knitted hosiery
- prosthetic garments for shaping oedematous soft tissue as well as for the improvement of wound healing and pain reduction after leg amputations

The advantages of flat knitted compression stockings:

- optimum, precise fit
- stronger effect through high working pressure during movement
- better grip and less pressure points and rubbing
- high stability ensures fit without folds and constrictions
- enhances subcutaneous microcirculation through massage effect



flat knitted with
smooth elastic flat seam

Stitch by stitch

Flat knitted compression hosiery is manufactured in a complex method stitch by stitch flat as a cloth. Only the flat knitting production process allows the number of stitches to be altered within the stitch course. Also, modifications can be made including but not limited to instep stitching, anatomical flexes, homogenous endings and slanted tops/ toes. After completion of the knitted fabric both endings are sewn together or better the stitches on both edges of the seam are linked with a single thread. This ensures optimum flexibility of the joints under compression.

Flat knitted compression products have less tension and are dimensionally stable. They do not lay into skin folds and prevent undesirable constrictions. The manufactured compression hosiery or compression segment models exactly the anatomy, even in case of vast proportional differences.

**For more information
on the flat knitted compression,
please see our
customer information
(Art.No.: 0561).**





COMPRESSANA FORTE pro

COMPRESSANA FORTE pro has a particularly supple knitted fabric that is characterised by its high wearing comfort and uniform appearance. It allows full mobility and is therefore ideal for the permanent/long-term care of unproblematic or fully treated oedema. Due to the relatively flat, fine-meshed knitted fabric, FORTE pro is considered to be rather comfortable, light and pleasant compared to stable variants. Thanks to the high UV absorption, the regeneration of young skin is supported. This property is particularly advantageous in scar and burn therapy.

CCL 1 light compression
2,4 - 2,8 kPa (18 - 21 mmHg)
Art.No. 135

CCL 2 moderate compression
3,1 - 4,3 kPa (23 - 32 mmHg)
Art.No. 235

CCL 3 firm compression
4,5 - 6,1 kPa (34 - 46 mmHg)
Art.No. 335

CCL 4 extra firm compression
> 6,5 kPa (> 49 mmHg)
Art.No. 435

Style


 **A-d**
A-d short
A-d extra short **Below-knee stockings**


 **A-d/H**
A-d/H short
A-d/H extra short **Below-knee stockings**
• Silicone band (3,5 cm)
• Softcontact band (3,5 cm)

 **A-f**
A-f/H **Half-thigh-high stockings**
• Silicone band (dots)
• Lace band
• Silicone dots w/ pattern
• Softcontact band

 **A-g**
A-g short
A-g extra short **Thigh-high stockings**

 **A-g/H**
A-g/H short
A-g/H extra short **Thigh-high stockings**
• Silicone band (dots)
• Lace band
• Silicone dots w/ pattern
• Softcontact band

 **A-g/T**
A-g/T short **Thigh-high stockings**
with waist attachments

 **A-t/L**
A-t/R **One-legged pantyhose**
with leg extension left or right

 **A-t**
A-t short
A-t extra short **Tights**
without adjustable waistband

- soft and smooth
- fine meshed
- effective penetration & working pressure



closed toe



open toe

Colours

sandel* silk choco* graphit* indigo*

black*

* only made to measure, only in CCL 1-3









COMPRESSANA FORTE stabil

Stable and strong: these are the properties of COMPRESSANA FORTE stabil. The high material thickness and strength ensure a stable wall effect and an unsurpassed depth effect. The air-permeable, porous, three-dimensional knitted surface develops a strong massage effect under movement.

The material thickness and stiffness of the knitted fabric ensures an even distribution of the support pressure in case of calibre jumps, e.g. before / after joints, skin furrows or skin flaps.



CCL 1	light compression 2,4 - 2,8 kPa (18 - 21 mmHg) Art.No. 125	CCL 2	moderate compression 3,1 - 4,3 kPa (23 - 32 mmHg) Art.No. 225	CCL 3	firm compression 4,5 - 6,1 kPa (34 - 46 mmHg) Art.No. 325	CCL 4	extra firm compression > 6,5 kPa (> 49 mmHg) Art.No. 425
--------------	---	--------------	--	--------------	--	--------------	---

Style	
 A-d A-d short A-d extra short	Below-knee stockings
 A-d/H A-d/H short A-d/H extra short	Below-knee stockings • Silicone band (3,5 cm) • Softcontact band (3,5 cm)
 A-f A-f/H	Half-thigh-high stockings • Silicone band (dots) • Lace band • Silicone dots w/ pattern • Softcontact band
 A-g A-g short A-g extra short	Thigh-high stockings
 A-g/H A-g/H short A-g/H extra short	Thigh-high stockings • Silicone band (dots) • Lace band • Silicone dots w/ pattern • Softcontact band
 A-g/T A-g/T short	Thigh-high stockings with waist attachments
 A-i/L A-i/R	One-legged pantyhose with leg extension left or right
 A-t A-t short A-t extra short	Tights without adjustable waistband

- **robust wall effect**
- **wide meshed, permeable (porous) massage knitting**
- **particularly effective penetration and working pressure**



closed toe



open toe

Colours

sandel*

silk

choco*

graphit*

indigo*

black*

* except Art.No. 425 (CCL4)



COMPRESSANA COTTON pro





COMPRESSANA COTTON pro wears naturally and is kind to the skin thanks to its high cotton content. Increased moisture absorption and excellent temperature regulation make it a reliable everyday companion. It allows full mobility with effective working pressure and depth. The soft and slippery material makes COMPRESSANA COTTON pro particularly recommended for **sensitive skin**, skin irritations or for patients who prefer cotton against the skin.





CCL 1 light compression
2,4 - 2,8 kPa (18 - 21 mmHg)
Art.No. 155

CCL 2 moderate compression
3,1 - 4,3 kPa (23 - 32 mmHg)
Art.No. 255

CCL 3 firm compression
4,5 - 6,1 kPa (34 - 46 mmHg)
Art.No. 355

Style

	A-d A-d short A-d extra short	Below-knee stockings
	A-d/H A-d/H short A-d/H extra short	Below-knee stockings • Silicone band (3,5 cm) • Softcontact band (3,5 cm)
	A-f A-f/H	Half-thigh-high stockings • Silicone band (dots) • Lace band • Silicone dots w/ pattern • Softcontact band
	A-g A-g short A-g extra short	Thigh-high stockings

	A-g/H A-g/H short A-g/H extra short	Thigh-high stockings • Silicone band (dots) • Lace band • Silicone dots w/ pattern • Softcontact band
	A-g/T A-g/T short	Thigh-high stockings with waist attachments
	A-t/L A-t/R	One-legged pantyhose with leg extension left or right
	A-t A-t short A-t extra short	Tights without adjustable waistband

- flat-knitted with natural cotton
- skin-friendly
- effective in penetration and working pressure



closed toe



open toe

Colours

silk


COMPRESSANA

SEGMENTS AND APPLICATION AREAS

of flat knitted products

Combinations possible within the framework of two-piece or multi-piece compression fittings for arm/hand and leg/foot


Stockings/Tights

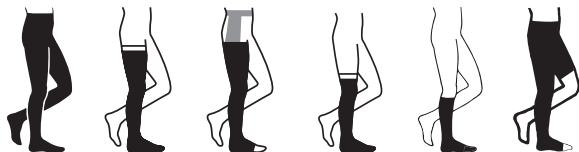


Stump fittings



Bermuda pants / Capri pants / Leggings

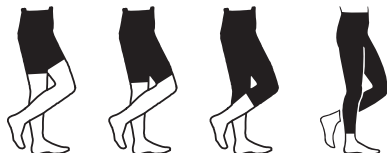

Leg segments

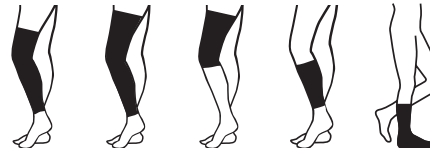

Sleeves / Arm segments



Toes / Fingers / Hands



Thorax / Head



COMPRESSANA SPECIAL MODELS

Special models compression



- | | | | |
|---|---|---|---|
| 1 Tights with different compression on each leg | 5 Compression stump stockings | 9 Open crotch | 13 3/4 or 1/1 silicone beaded band (dots) at the inside |
| 2 Gentlemen's panty with fly | 6 Pieces of silicone beaded band (dots) | 10 Hook closure / Velcro fastener / Reinforced hook closure | 14 Diagonal foot |
| 3 Stoma opening | 7 Waist attachment | 11 Trunk with increased compression | 15 Bermuda pants / Capri pants / Leggings |
| 4 Oblique slanted border / Overheight at "G" | 8 Anatomically flexed form at "cE" (30° or 50°) | 12 90° Foot | 16 One-legged pantyhose |

COMPRESSANA COMPRESSION PADS

In compression therapy, pressure pads are used on the palms/back of the hands or on legs in the ankle pits to pad concave areas, to create the most even pressure possible or to increase pressure in a targeted manner. Pads are mainly used in oedema therapy to promote microcirculation, increase lymphatic drainage, resorb oedema, remove deposits and loosen fibrosis. Also in scar treatment, compression promotes tissue healing, reduction of

scar thickness and reduction of itching and redness of the skin. In addition to silicone sheets and plates, pads are also used to increase or optimise pressure.

COMPRESSANA pads offer individual fitting options with the best possible fitting, optimal wearing comfort and skin tolerance!

For Bisgaard area

Suitable for both left and right sides of the achilles tendon and for the curve (indent) below ankle, covered with fabric.

Art.No. 0123 LA/RI

Art.No. 0124 RA/LI



Compression pad – short

For the back and palm of the hand. For treating oedema and for cone shaped parts of the hand.

Art.No. 0111



Compression pad – long

For the hand. Enables additional pressure. Wrist overlapping.

Art.No. 0159



Compression pad – silicone

Naked or covered with Charmeuse.

Art.No. (naked)

Art.Nro (covered)



Possible applications under compression stockings, segments, bandages and dressings:

- Loosely inserted (individual placement by the customer) or fixation with COMPRESSANA ULCER SYSTEM or COMPRESSANA INTRA.
- Sewn in (glued or sewn): Is glued into the stocking/bandage with silicone glue at the chosen location after being marked by the customer and then covered with lining fabric so that the location remains elastic.
- Slipped into pockets made of lining fabric: Pocket is sewn on the inside of the sock/segment after marking by the customer and pad can be slipped in.

COMPRESSANA SLEEVES

CCL 1 2,4 - 2,8 kPa
(18 - 21 mmHg)

CCL 2 3,1 - 4,3 kPa
(23 - 32 mmHg)

CCL 3 4,5 - 6,1 kPa
(34 - 45 mmHg)

Style



C-G

Sleeve





C-H

Sleeve
with shoulder holder



C-H

Sleeve
with bra holder

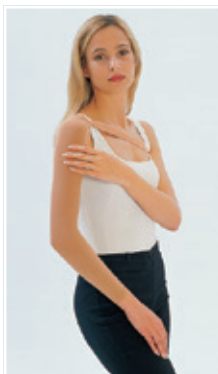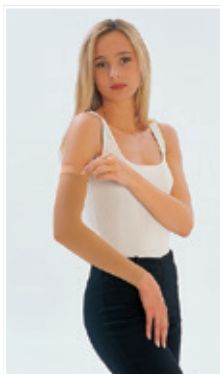

COMPRESSANA

HAND- & TOE PARTS

Style



Finger support
open or closed



Gauntlet until c_1
with thumb stub



Glove until c_1 ,
with closed fingers or finger stubs
Glove long until d or e,
with closed fingers or finger stubs



Style



Toe part
with open toes / closed toes



Colours

sandel

silk

choco

graphit

indigo

black



INTRA toe sock,
closed toe



INTRA toe sock
COMPRESSIONA FORTE



INTRA half toe sock,
open toe



INTRA half toe sock with
COMPRESSIONA FORTE



Base layer sock
with silver



Base layer stocking
with silver



Base layer stocking
with COMPRESSIONA FORTE



Underliner
COMPRESSIONA ULCER INTRA

COMPRESSANA INTRA

Base layer and fixing system

Base layer and fixing system for modular wound and skin treatment - scope of application:

- Fixation of wound dressings for the treatment of small skin injuries.
- Slipping-free fixation of truss pads, inlays to protect against pressure and pads for the treatment of medically and anatomically complex situations on the foot, ankle or lower leg.
- In combination with flat or round knitted compression stockings, pressure truss pads and wound dressings for the modular ulcer-cruris treatment.
- For patients with sensitive skin to put on underneath compression stockings and supports.
- Due to high flexibility suitable for wearing in preference to socks/below-knee stockings in presence of tenderness, deformation, adhesion, large leg and foot circumferences etc
- Toe parts (toe support or toe protection) ideal for the supporting treatment of the gaps between the toes or for the combination with compression or supporting stockings with open toe.
- Can be worn under socks/socks in case of foot perspiration, foot odour, skin or fungal disease.
- Silver fibres are used for sensitive, stressed and inflamed skin and also with chronic skin diseases. Silver ions develop when the fibre gets in contact with the skin and react with the protein of the germ and thereby exhibit an antimicrobial effect. Silver reduces the odour formation. Furthermore its outstanding conductivity helps thermo regulation and reduces static.

Product	Colour	Size	PZN	Art.No.	RRP*
INTRA Toe sock, closed toe with cotton & Coolmax with forefoot support to relieve symptoms of hallux valgus	white	One-Size	09733715	0090	12,90 €
INTRA Half toe sock, silver	white	One-Size	09210897	0095	7,50 €
INTRA Base layer sock, SkinProtect®Silver X-Static	silver	One-Size	09210880	0094	18,- €
INTRA Base layer stocking, SkinProtect®Silver X-Static	silver	One-Size	09210905	0096	19,90 €
INTRA Base layer stocking extra wide, SkinProtect®Silver X-Static	silver	Calf circumference > 50cm and large foot sizes	07217116	0086	22,- €
INTRA Underliner CCL1	silk	I – VI	depending on size	0183	49,- €

* approx. recommended retail price

COMPRESSANA MASTER

Donning and doffing aid



Donnig

In a sitting position, without bending down and without exerting force by pre-stretching the stocking material.



Doffing


Removing the stocking is made easier by a retaining element that secures the stocking for removal and a handle that „extends“ the arms.

With a little practice, COMPRESSANA MASTER makes it possible to put on and take off stockings • effortlessly and simply • in a sitting position • without strenuous bending • without exertion.

The pre-stretching of the stocking material reduces friction on the skin and protects the skin. In addition, the knitted fabric of the stockings is protected from damage.

COMPRESSANA Master is suitable for all people who prefer it lighter or who have special circumstances or situations, e.g. limited mobility, etc.

COMPRESSANA Master makes it easier to start compression therapy and ensures that the stockings are put on correctly. The doctor recommends or prescribes donning and doffing aids in the case of illnesses or disabilities with significant functional disorders of the finger, hand and arm joints, also in the case of limited gripping function, which make it impossible to put on stockings without aids. Compressana MASTER can be disinfected. Wipe disinfection with a surface disinfectant can be carried out.

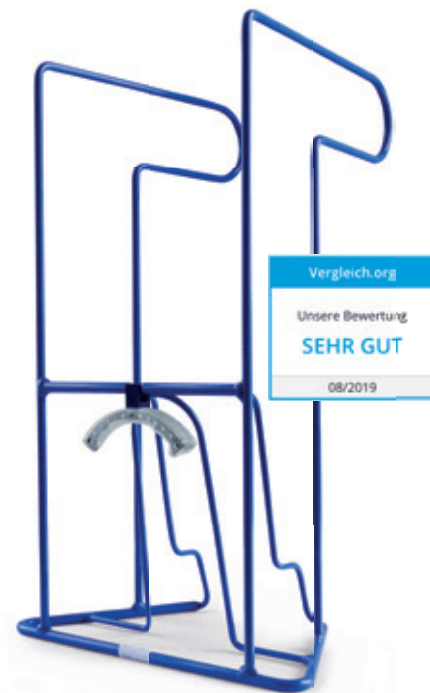

Product	Art.No.	RRP*
MASTER PZN: 09717627 HMV-Nr. 02.40.01.3028	0560	85,- €
MASTER XL Ankle circumference cB > 26 cm PZN: 07140218 HMV-Nr. 02.40.01.3028	0565	85,- €
Surface disinfection wipe, non-alcoholic, 10er-Set PZN: 16730746	0614	6,90 €

COMPRESSANA SNAPPER

Donning Aid




Donning and Doffing according to the Snapper concept is quick and requires little effort. This is ensured by the special construction and the gliding ability of the Snapper donning facilitator.

It is also advantageous to use stocking gloves with grippy palms, as recommended in the stocking manufacturers' instructions for use. These are available as accessories.

COMPRESSANA Snapper protects the skin from injuries and stocking stitches from damage due to mechanical effects, e.g. friction, finger/toe nails, rings or jewellery, when putting on and

taking off, especially in combination with Top-Grip stocking gloves. The stockings thus retain their therapeutic effect for longer.

Compressana Snapper makes it easier to start compression therapy and ensures that stockings are put on correctly. Compressana Snapper is suitable for all people who prefer it lighter or who have special circumstances or life situations, e.g. limited mobility.

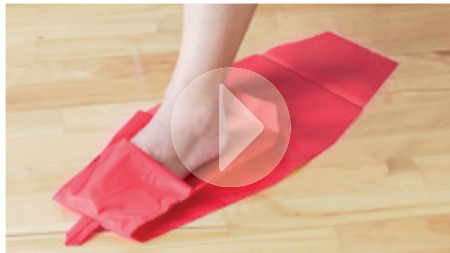

We make it easy for you to get dressed and undressed:

Video instructions at www.compressana.de or scan QR code with smartphone/tablet.



SNAPPER/MASTER
experience live

**[www.youtube.de/
compressana](http://www.youtube.de/compressana)**

Product	Art.No.	RRP*
SNAPPER PZN: 12399415 HMV-Nr: 02.40.01.3054	0089	36,00 €
Surface disinfection wipe, non-alcoholic, 10er-Set PZN: 16730746	0614	6,90 €

* approx. recommended retail price



COMPRESSANA EASY – Donning Aid Leg & Donning Aid Arm

COMPRESSANA EASY, the simplest and cheapest donning aid and an alternative to expensive racks. The particularly slippery material goes around the foot and leg, enables donning of stockings even with high pressure without any problems. Equally suitable for open and closed toe. COMPRESSANA offers also a simplification for the donning of compression sleeves. The particularly slippery material enables donning of sleeves even with high pressure without any problems.

- easy to handle
- sturdy
- perfect for mobile nursing services

Product	Art.No.	RRP*
EASY Leg PZN: 05704792 HMV-Nr. 02.40.01.3041	0098	11,- €
EASY Arm One size HMV-Nr. 02.40.01.3040	0120	19,50 €

COMPRESSANA Sliding Aid

The sliding sock is pulled over the forefoot; its slippery material makes it easy to pull the compression stocking onto the foot. The sliding sock is then simply removed through the toe opening. When the toe is closed, the sliding sock can be removed with the toe flap pointing back towards the heel.

Product	Art.No.	RRP*
EASY Sliding Aid PZN: 01556254	0100	3,- €

COMPRESSANA SPECIAL GLOVES

TIP The use of gloves makes it much easier to put on stockings and trousers and protects the stocking stitches from damage by fingernails or jewellery.



COMPRESSANA SUPER-GRIP

Extra fine, skin-friendly and breathable textile gloves Extra grip, perfect fit, seamless, better fingertip feeling.



COMPRESSANA TOP-GRIP

Breathable, sensitive and durable textile gloves made of nylon with nitrile coating – seamless, hardwearing.



COMPRESSANA GRIP

Special gloves made of natural rubber (latex) with non-slip palm, very good dexterity and cotton lined.

Product	Art.No.	RRP*
SUPER-GRIP Gr.II (S) until 8 cm hand-width* PZN: 08719558	0112	6,00 €
SUPER-GRIP Gr.III (M) until 9 cm hand-width* PZN: 08720573	0112	6,00 €
SUPER-GRIP Gr.IV (L) until 10 cm hand-width* PZN: 08720596	0112	6,00 €
SUPER-GRIP Gr.V (XL) until 11 cm hand-width* PZN: 12470018	0112	6,00 €

Product	Art.No.	RRP*
TOP-GRIP Gr.II (S) until 8 cm handwidth* PZN: 12469972	0108	4,50 €
TOP-GRIP Gr.III (M) until 9 cm handwidth* PZN: 12469989	0108	4,50 €
TOP-GRIP Gr.IV (L) until 10 cm handwidth* PZN: 12469995	0108	4,50 €
TOP-GRIP Gr.V (XL) until 11 cm handwidth* PZN: 12470001	0108	4,50 €

Product	Art.No.	RRP*
GRIP Gr.II (S) until 7,7 cm handwidth* PZN: 03415612	0102	3,50 €
GRIP Gr.III (M) 7,8 to 8,7 cm handwidth* PZN: 03415635	0102	3,50 €
GRIP Gr.IV (L) 8,8 to 9,8 cm handwidth* PZN: 03415658	0102	3,50 €

* The size is determined by measuring the hand width at the finger baseline

* approx. recommended retail price

COMPRESSANA FIXATION

The right fixation for every stocking

Integrated fixing aids, silicone-coated bands, internal bands, hip fixation, bodice, shoulder strap or bolero are available for fixing in the leg and arm area.


The attractive and the neutral silicone bands of the highest quality are breathable, comfortable to wear and adapt perfectly to the leg. Choose according to your needs and preferences, by design and type of coating.



Silicone beaded band
(dots)



Lace band

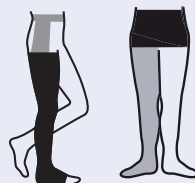

Hip and body attachment



Silicone dots w/ pattern



Softcontact band



Further information on
Bands can be found in the
download area at
www.compressana.de

Skin adhesive for effortless hold

The skin-friendly adhesive lotion, which can be removed without leaving any residue, gives stockings, bandages and other textiles a secure and long-lasting hold. Simply spread the adhesive lotion on the skin, press the stocking or bandage against it - and you're done. After wearing, it can be removed from the skin within seconds with water or a damp cloth.



Product	Art.No.	RRP*
Roll-On 60ml PZN: 05704786 HMV-Nr. 17.06.07.0011	0116	13,50 €

COMPRESSANA LYMPHATIC CUSHION

Positioning and supporting aid for arm and hand

Comfort, hygiene and maximum effectivity – also during sleep and at work

The healing of lymphatic oedema and serious injuries of the arm and hand is supported by an elevated resting position. Swelling is reduced or even prevented. Improves back flow of the blood to the heart as well as the removal of the lymph fluid. The COMPRESSANA lymphatic cushion is a valuable treatment aid for "complex physical decongestive therapy" (CPDT). It can also be used for pain treatment and wound management. Patients with lymphatic oedema, lip oedema or burns/scalding quite often suffer from bacterial infections (erysipiel, abscesses, pus-producing infections and staphylococcus). These infections cause additional swelling, itching, redness of the skin and pain. Therefore, the armrest surface of the COMPRESSANA lymphatic cushion is coated with a silver lining. The silver lining has a bacteriostatic

effect, keeps the natural balance of the skin flora and can help to reduce irritating side effects of infections. Additionally, it protects the affected skin parts against contracting further infections.



Product	Art.No.	RRP*
LYMPHATIC CUSHION Gr.I ≤ 30 cm Size is determined by the circumference of the forearm PZN: 09717656	0550	179,95 €
LYMPHATIC CUSHION Gr.II > 30 cm Size is determined by the circumference of the forearm PZN: 09731455	0550	179,95 €

* approx. recommended retail price

COMPRESSANA SKIN CARE



For care of skin under treatment with
compression garments

Optimal care products for compression therapy

To ensure successful compression therapy for vascular diseases, oedemas etc., a healthy, elastic and resistant skin is of great importance. The development of the products was carried out in close cooperation with renowned dermatological cosmetics manufacturers. We exclusively use formulas with ingredients that have successfully proven themselves in practise for many years in accompanying therapy at clinics and practitioners. With the product line COMPRESSANA SKIN CARE COMPRESSANA offers care products especially adapted to the needs of compression therapy for prophylaxis and skin protection.

For more information
on Skin Care, please
see our customer
information
(Art.No.: 0737).



Further information on Skin Care & Compression can be
found in the download area at www.compressana.de

COMPRESSANA ANTI-ITCHING SPRAY

The active ingredient polidocanol soothes the skin and can prevent itching for several hours. The spray provides moisture and has an antimicrobial effect against excessive bacteria and germ formation. Inflammation caused by scratching is prevented.

It can be used several times a day on affected skin areas or directly on support and compression stockings, is absorbed without residue and does not leave any stains. The spray can be used independently or in addition to the cream or lotion.

Ingredients: Aqua, Butylene Glycol, Laureth-9, Chlorhexidine Digluconate



Product	Art.No.	RRP*
ANTI-ITCHING SPRAY Sprühflasche 150 ml, PZN: 15618510	0583	15,- €
ANTI-ITCHING SPRAY Sprühflasche 50 ml, PZN: 15618496	0580	10,- €

COMPRESSANA TONIC

- **Vitalizes + strengthens the veins**
- **Refreshes + cools the legs**
- **Nourishes + moisturizes**
- **Medicinal plant extracts from horse chestnuts, sage, comfrey and witch hazel**
- **Care complex with evening primrose and apricot kernel oil**

The tonic combines proven venous agents and sustainable care substances to create a unique and well-tolerated composition. In combination with support and compression stockings, blood circulation and microcirculation can be improved. Uncomplicated use! Can also be sprayed directly onto the stockings.

Ingredients: Aqua, Alcohol Denat, Glycerin, PEG-40 Hydrogenated Castor Oil, Sodium Lactate, Menthol, Lactic Acid, Hamamelis, Virginiana Leaf Extract, Litsea Cubeba Fruit Oil, Oenothera Biennis Oil, Prunus Armeniaca Kernel Oil, Symphytum Officinale Root Extract, Aesculus Hippocastanum Seed Extract, Camphor, Juniperus Oxycedrus Wood Oil, Linalool, Geraniol



COMPRESSANA VLIES

Practical care tissues
for skin and band

We make compression treatment easy and comfortable.

Compressana VLIES is used to care for areas of skin under compression treatment – even in the area of the band – and to clean the silicone coating if necessary. The fluid contained in the care tissue protects sensitive skin from irritation, soothes itching, moisturises and removes flaky skin. Skin irritations are prevented. It also ensures a better hold and fit of support stockings and prevents slipping.

Ingredients: Aqua, Caprylic/Capric Triglyceride, Pentylene Glycol, Laureth-9, PEG-40 Hydrogenated Castor Oil



Product	Art.No.	RRP*
TONIC Spray 200 ml, PZN: 13360898	0953	15,- €
TONIC Spray 50 ml, PZN: 13360881	0952	6,- €

Product	Art.No.	RRP*
VLIES Care tissues in sachets, set of 10 PZN: 12469162	0954	6,40 €

COMPRESSANA SKIN PROTECT

Intensive care and repair creme

Skin regeneration and acute care of stressed, sensitive areas of the skin, even in cases of venous disorders. The antimicrobial effect of silver reduces the risk of infection. The repair complex can counter inflammation, irritation, etc. and promotes skin renewal.

For regular use, especially after wearing support and compression stockings, recommended especially on irritated skin areas. Also suitable for neurodermatitis and diabetics.

Ingredients: Aqua, caprylic/capric Triglyceride, Panthenol, Glycerin, Pentylene Glycol, Polyglyceryl-3 Methylglucose Distearate, Zinc Oxide, Glyceryl Stearate, Stearyl Alcohol, Tocopherol, Silver, Citric Acid



Product	Art.No.	RRP*
SKIN PROTECT Tube 75 ml, PZN: 15618533	0757	11,- €
SKIN PROTECT Mini-tube 10 ml, PZN: 15618527	0756	6,- €

COMPRESSANA VEIN BALM

The vein balm protects and nourishes the skin and gives a pleasant and fresh feeling before and after wearing compression or support stockings.

Absorbs quickly and without residue. Selected natural ingredients (horse chestnut, arnica fluid extract and essential oils such as eucalyptus, camphor and rosemary) provide additional help and skin care for vein problems.

Ingredients: Aqua, Paraffinum Liquidum, Isopropyl Alcohol, Caprylic/Capric Triglyceride, Glycerin, Urea, Cetearyl Alcohol, Ammonium Acryloyldimethyltaurate/Vp Copolymer, Trilaureth-4 Phosphate, Polyglyceryl-2-Sesquiosseate, Eucalyptus Globulus Oil, Dimethicone, Camphor, Arnica Montana Extract, Aesculus Hippocastanum Seed Extract, Phenoxyethanol, Ethylhexylglycerin.



Product	Art.No.	RRP*
VEIN BALM Dash 125 ml, PZN: 04139120	0104	11,- €
VEIN BALM Mini-tube 50 ml, PZN: 15390490	0113	450 €

COMPRESSANA DERMA LOTION

Highly intensive emollient and skin protection for varicose veins, lipoedema, lymphatic oedema and skin diseases.

The lotion provides intensive and long lasting moisturisation. Protects the skin, keeps it supple and resistant and prevents irritations and itching. For regular use after wearing compression, support and functional stockings.

Ingredients: Aqua, Isopropyl Palmitate, Caprylic/Capric Triglyceride, Butylene Glycol, Urea, Diisostearyl Polyglyceryl-3 Dimer Dilinoleate, Dimethicone and Dimethiconol, Magnesium Sulfate, Hydrogenated Castor Oil, Cera Alba, Phenylpropanol, Tocopherol, Bisabolol, Backhousia Citriodora Leaf Extract, Citral



Product	Art.No.	RRP*
DERMA LOTION Bottle 500 ml, PZN: 07240115	0950	22,- €
DERMA LOTION Bottle 125 ml, PZN: 07240121	0925	11,- €

COMPRESSANA HYDRO LOTION

For improved tolerance and better grip of stockings. Also suitable for application in the area of the grip top and silicone adhesive edge, before and after wearing compression or support stockings.

Acute care against itching and for protection against irritations. Calms and soothes sensitive skin from mechanical irritations and rubbing. Enhances the skin regeneration and provides long lasting moisturisation.

Ingredients: Aqua, Caprylic/Capric Triglyceride, Glycerin, Pentylene Glycol, Polyglyceryl-3 Methylglucose Distearate, Laureth-9, Stearyl Alcohol, Glyceryl Stearate, Zinc Oxide, Tocopherol, Citric Acid, Xanthan Gum



Product	Art.No.	RRP*
HYDRO LOTION Bottle 200 ml, PZN: 07239595	0920	17,- €
HYDRO LOTION Airless-Spray 50 ml, PZN: 12671685	0951	9,50 €

COMPRESSANA ACCESSORIES

FOR CARE, WASHING AND STORAGE



Shoehorn long

This shoehorn makes it easier you to put on your shoes.

- Standing, with restricted mobility.
- with hook and practical hanging hole

Pocket nail file

Nail file in practical pocket style. Always ready to use and quickly to hand. Also ideal for on the go. To protect your stockings and for strong smooth nails, we recommend weekly nail care.



Double foot scraper

Double-sided callus file for gentle foot care that is kind to the skin. Pre-treat the callus with the coarse rubbing surface and smooth it with the fine side. Then apply SKIN PROTECT or a rich foot cream/callus balm. We recommend regular use to protect your stockings and for healthy, beautiful feet.



Product	Art.No.	RRP*
Shoehorn long approx. 65 cm PZN: 08720805	0935	3,- €

Product	Art.No.	RRP*
Pocket nail file PZN: 12672319	0955	3,50 €

Product	Art.No.	RRP*
Double foot scraper, waterresistant, PZN: 11182398	0956	3,- €

Special bag – Tip for travelling and storage

Sharp edged or pointed objects can damage the knitting of your valuable stockings and tights. In this special bag they can be kept protected and transported. Helps you keep order at home and on journeys.



Product	Art.No.	RRP*
Special bag for support and compression hosiery, 23 x 28 cm PZN: 09211000	0103	23,50 €

Tip for the perfect care of the stockings

The special detergent COMPRESSANA CLEAN is suitable for elastic and high-quality textiles. Also for wool, silk and synthetic high-tech garments.



Clothes pegs – made of untreated beech wood



Product	Art.No.	RRP*
CLEAN Special detergent Tube 250 ml, PZN: 09210503	0121	4,50 €
CLEAN Special detergent Travel tube 50 ml, PZN: 13360906	0085	1,95 €
Wäscheklammern aus Buchenholz, 6 Stck, 90 mm	0708	1,95 €

Laundry bag

Protects your stockings and tights during machine wash. Quick and easy to fill by means of zipper.



Silver laundry bag

With pure silver fabric (0% nano particles) to combat unpleasant smelling laundry, even at low temperatures.

Product	Art.No.	RRP*
Laundry bag with zipper, 100% Polyester PZN: 00065910	1007	4,50 €
Silver Laundry bag with pure silver fabric, 50% PES, 50% PA-silver coated	0891	26,95 €

COMPRESSANA INSHOE-SOCKS & STEPS



Inshoe-Socks with wool



Inshoe-Socks with cotton



Steps with cotton

Fashionable short shaft socks. Slipped over compression or support stockings/tights, the inshoe-socks provide for warm feet during the cold season and protect the meshes against damage caused by footwear or little stones.

- Footlets with highest wearing comfort
- Flat non-constricting elastic band at the cuff
- Particularly flexible and soft
- More comfortable than barefooted in the shoe
- Protect valuable hosiery against damage in the foot part
- Breathable, not bulky
- Create a healthy and pleasant climate in the shoe
- Can help to avoid blisters.

Product	Colour	Size	Material	Art.No.	RRP*
Inshoe-Socks with wool	anthrazit	35–38 39–42	80 % wool, 20 % PA	1003	8,- €
Inshoe-Socks with cotton	black, natur	43–45	98 % cotton, 2 % PA	1004	7,50 €
STEPS with cotton	silk	35–38, 39–42	60 % cotton, 40 % PA	1005	4,- €



COMPRESSANA STANDS FOR HIGH QUALITY

Individuality: In our range, you will always find the right sock in all different styles, colours and equipment or band variants.

Safety: Our tested sizing system with six particularly tolerant standard sizes in three different lengths or exact custom-made sizes ensures optimal relief through a precisely defined pressure curve.

Guaranteed value: Fine yarns and masterful manufacturing make for genuine quality stockings with pleasant wearing comfort and a beautiful stitch pattern.

Your quality and fit guarantee: We will replace our compression products free of charge for 6 months from the date of purchase. The prerequisite is proper handling according to our instructions for use and care and the correct recording of body measurements or the correct determination of size.

Goodwill: Despite proper use, damage to fine meshes due to unfortunate circumstances, carelessness or external influences - who hasn't experienced this? Instead of „comprehensive insurance“, we offer a goodwill arrangement for the „worst case scenario“: replacement with new goods at a goodwill price within 6 months.



ASSORTMENT WITH SYSTEM

Sustainability: COMPRESSANA

compression stockings are harmless to human ecology and health and have been awarded the Oeko-Tex Standard 100 quality seal.

Compressana does not use additives such as silicone or other softeners in the knitted fabric. All Compressana compression stockings are manufactured according to the highest quality standards, in an environmentally friendly and resource-saving manner in one of the most modern knitting factories in Germany.



		NOVA	LEGANZA	DIAMONDS	SENSITIV	LINEA	LINEA COTTON SPORT COTTON	COTTON	NEU	FREE	FORTE	ULCER SYSTEM/ INTRA	COTTON pro	FORTE pro	FORTE stabil
white			S/M	S	S/M										
facing grey							S/M								
sandel			S/M	S	S/M					S/M	S/M			M	M
silk		S/M	S/M	S	S/M	S/M	S/M	S/M		S/M	S/M	S/M	M	M	M
perle		S/M	S/M	S	S/M										
choco			S/M	S	S/M	S/M	S/M							M	M
graphit			S/M	S	S/M	S/M	S/M			S/M	S/M			M	M
indigo			S/M	S	S/M	S/M	S/M			S/M	S/M			M	M
black		S/M	S/M	S	S/M	S/M	S/M	S/M		S/M	S/M			M	M
Gradient Colours															
rising indigo			S/M												
rising graphit			S/M												
Colour Accents															
			S/M	S	S/M										
round knitted												flat knitted			
closed toe	CCL 1	180	160	168	195	196	156	150	170	130	183 ³	155	135	125	
	CCL 2	280	260	268	295	296	256	250	270	230		255	235	225	
	CCL 3									330		355	335	325	
	CCL 4												435	425	
open toe	CCL 1	181	161		190			150	170	130		155	135	125	
	CCL 2	281	261		290			250	270	230		255	235	225	
	CCL 3									330	333 ³	355	335	325	
	CCL 4												435	425	
Style					A-d	A-d			A-d A-g/H		A-d	Series in A-d CCL 1-2	Series in A-d CCL 1-2		
Borders Style A-G															
Trikotborder		-	-	-	S/M	-	-	S/M	-	S/M	-	-	-	-	-
Lace band		S/M	S/M	-	S/M	-	-	S/M	S/M	S/M	-	M	M	M	M
Softcontact band		-	S/M	S	S/M	-	-	S/M	S/M	S/M	-	M	M	M	M
Silicone beaded band dots		S/M	S/M	-	S/M	-	-	S/M	S/M	S/M	-	M	M	M	M
Silicone dots w/ pattern		S/M	S/M	-	S/M	-	-	S/M	S/M	S/M	-	M	M	M	M
Borders Style A-D															
Silicone beaded band dots (3,5cm)		S/M	S/M	S	S/M	S/M	S/M	S/M	S/M	S/M	-	S ¹ /M	S ¹ /M	M	
Softcontact band (3,5cm)		-	S/M	S	S/M	-	S/M	S/M	S/M	S/M	-	M	M	M	M
Band 5 cm		-	-	-	-	-	-	M ²	-	M ²	-	M ²	M ²	M ²	M ²
S=Standard size, M=Made by measure ¹ (Standard size CCL silk) ² (optionally Silicone beaded band, Lace band, Silicone dots w/ pattern or Softcontact band) ³ Overstocking open, INTRA Underliner closed															



COMPRESSANA APP

Download the COMPRESSANA App on your tablet or smartphone. Use the new COLOURS 2021 consulting tool for easy colour selection of accessories and compression stockings. In addition, you will find selected leaflets and flyers and constantly supplemented and expanded information on our products.



App in German language available at Google Play Store and App Store.



COMPRESSANA GmbH
Böhmerwaldstraße 3 · 93073 Neutraubling · Germany
Tel.: 09401 9226-0 · Fax: 09401 9226-20
info@compressana.de · www.compressana.de