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Medical compression stockings exert a firm pressure on the parts of the body which are to be treated. Of particular importance for the medical effectiveness, correct fit and wearing comfort are both the accurate adaptation of the two-way stretch knitted fabric and the appropriate application free from creases. A good briefing concerning the handling, some practise and reasonable diligence help to reach thereby the necessary routine for daily application. Compression products consist of very flexible materials. They should be donned (without overstretching) instead of just be put on.

Closed toe

- 1 Turn the stocking inside out up to the heel. The footpart remains in the stocking. Expand the foot by opening both hands and pull the stocking over the foot up to the heel.
- 2 a) The stocking turns back by pushing in the leg. This forms a double fold in the middle of the foot.
b) Seize with two fingers into the double fold and lift the stocking over the heel.
If the heel should not sit accurately or the stocking is twisted, repeat this procedure again.
- 3 Push back the stocking to the ankle and turn it back the right way without tension on upper and lower leg. Alternatively you can slip on the stocking step by step by using COMPRESSIONA rubber gloves.
- 4 In case of tights begin with steps 1.-3. After having pulled both leg parts up until just below the knee, ease them up both legs to the crotch, as described in 3. Only now you should pull the trouser up to your waist. The gusset should sit accurately in the crotch.

Open toe

- 1 Use the enclosed COMPRESSIONA pull-on aid as demonstrated in the illustration.
- 2 Turn the stocking inside out around the half of the overall length and slip in with your foot as far as possible by expanding the foot opening both hands. Due to the grip at the palm of the COMPRESSIONA rubber gloves the stocking can be raised easier.
- 3 After pulling up the stockings remove the pull-on aid by turning back the front foot part towards the ankle. Finally pull the front foot part back over the foot and smooth down the fabric to the toe section.
- 4 In the case of tights follow the procedure described above.

Advice for putting on the stockings

COMPRESSIONA GRIP or SUPER-GRIP special rubber gloves simplify the donning of stockings considerably. Due to the grip at the palm and a special surface texture the stockings can be raised easier. With these special gloves you can easily correct the fit of the stockings later or eliminate folds and twists. In addition the stockings are protected against damage (sharp edged fingernails or jewelry).

Putting on the stockings is much easier by using talcum powder on moist feet and legs.

Before donning the stocking, examine it for damage and follow any existing marking "left" or "right".

Make sure that the heel part fits properly. For a loose fit at the tip of the foot pull the stocking a little towards the front.

Never turn the top end of the stockings inside out. If you have pulled the stockings up too far push them back by using GRIP gloves until the mesh is even and there are no folds in the hollow of the knee.

Sticking out threads or loops on the inner surface are technically conditioned and should not be cut off, since otherwise holes and ladders might develop.

Your specialist retailer or dispensing chemist will gladly instruct you in handling and care of your COMPRESSIONA stockings.

Notes for adhesive bands / silicone border

Never use the grip top to pull up the stockings. When wearing stockings with silicone border ensure that the legs are free from ointment, powder, oil and if necessary depilated.

Even if used properly scalls, sweat or oils build up on the silicone coating and reduce the adhesion.

Should the stocking slip while wearing or you want to change the position of the silicone strips (for improvement of wearing comfort during the day) clean the silicone surfaces with the COMPRESSIONA CARING & CLEANING CLOTH.

Videos of several donning methods and usage of COMPRESSIONA SNAPPER, MASTER or EASY can be found under:
www.youtube.com/compressiona

The hold of silicone border can be limited by strong hairiness, very soft hypodermis or large extent increases of the thigh.

If necessary use additional attachments or alternatively fix the stockings underneath the silicone border with the COMPRESSANA hypoallergenic adhesive lotion.

COMPRESSANA HYDRO LOTION or COMPRESSANA CARING & CLEANING CLOTH improve the grip and can be applied on to skin in the region of the silicone border. They protect sensitive skin, prevent skin irritation and enhance skin regeneration while providing long lasting moisturisation.

Application

Medical compression stockings may not be tightened or worn with open wounds. If compression stockings are prescribed with open wounds, a professional sore treatment and cover has to be provided.

If necessary, carry out a skin inspection and check whether skin damage or pressure marks are present that require diagnostic confirmation.

Usually compression stockings are not to be worn during sleep or when lying long times, unless the doctor recommended or ordered explicitly a special night supply.

Unpleasant folds in the hollow of the knee can occur while sitting in strongly bent positions or kneeling for long periods. If necessary clarify the alternative supply of knee stockings with your doctor or put a light compress inside the sock into the hollow of the knee.

If swellings are present before the donning of the stocking, try to wipe them off in direction of the heart while holding or putting up your legs (for example lying on the back).

In case of unusually strong swollen legs possibly with skin furrows, in particular swollen foot backs or toes, the suspicion of lymphoedema arises.

Hereby complex physical decongestion is advisable. This supports compression therapy by manual lymphatic drainage, exercise therapy and skin care.

The handling and care take place as per advice from the doctor or medical personnel.

If there are difficulties with putting on the stockings, pressure or tightness feelings, e.g. diabetes or foot illnesses or during strong perspiration within the toe range, the open toe is to be preferred.

For therapeutic reasons it is recommended to put on the stockings as soon as you get up in the morning. At this time your legs are least congestion and at their smallest.

To ensure therapy success a regular daily application and procurement at the latest after 6 months is highly recommended, if the disease requires it. Additional exercises, gymnastics and physiotherapy exercises will support the soothing effect of your COMPRESSANA stockings.

Useful advice

Have your compression stockings checked regularly latest after 6 months at your dispensing chemist or specialist retailer. This ensures the quality and medical effect of the products.

For sensitive skin or striking irritations we recommend compression products with **cotton inside** like COMPRESSANA COTTON or LINEA COTTON.

Use **COMPRESSANA hypoallergenic adhesive lotion** for fixation and protection against slippage.

The adhesive lotion can be applied on all parts of the skin which is covered by the stockings. Do not use within the range of the silicone coating.

The products can be used in combination with **skin care preparations**, as long as the preparation is absorbed completely.

After wearing compression stockings arrange for a **moisturising skin care** to prevent dryness and itching.

The **COMPRESSANA vein balsam** nourishes and protects your legs and soaks in completely. Its application is recommended before and after wearing compression stockings.

COMPRESSANA TONIC provides pleasant refreshment and moisture for the legs. It can be sprayed directly onto the stockings.

COMPRESSANA inshoe socks worn over compression stockings provide warm feet and protect stockings and meshes against damage.

You can obtain selected COMPRESSANA care or supplementary products from your pharmacy, medical supply store or on the Internet at **www.compressana.de**

Donning- and doffing aids

The COMPRESSANA donning aids facilitate donning and doffing of compression hosiery. Friction on the skin is reduced and skin as well as the fabric become protected against damages. COMPRESSANA donning aids can be prescribed by doctors.

COMPRESSANA SNAPPER donning aid

The donning aid allows fast, simple and comfortable donning without exertion of force, even for moist or lotioned legs.

COMPRESSANA MASTER donning and doffing aid

In case of restricted mobility or disfunctions of fingers/hand the special metal construction of COMPRESSANA MASTER facilitates hassle free donning and doffing of stockings – without strenuous bending, while sitting and without exertion of force.

COMPRESSANA EASY donning aid

The simple and cheap donning aid. The particularly sliding cloth enables donning of stockings without friction resistance. Also simplifies donning in case of wound dressings or bandages at foot or leg.

Material composition

For precise information/specifications please see the label sewn in your compression stockings. All yarns used are resistant to sunlight and natural rubber. COMPRESSANA compression stockings are free of latex and rubber.

Durability/Storability

Store at room temperature and in a dry place. Avoid heat and moisture. In the case of series productions, the maximum storage duration is 36 months. A 6-month service life of the compression garments follows that period. The time of use of the medical product is printed onto the box label with an hour glass symbol.

Tailor-made items are products that are designed for immediate use. Since the body size and dimensions may change due to the progression of an illness or specific personal circumstances, tailor-made products are designed for 6 months of use. After 6 months of use, a renewed measuring of the body dimensions by a medical specialist becomes necessary.

In case of item damage please contact your specialist retailer or dispensing chemist. Do not attempt to mend stockings yourself as this could affect the medical effect.

Warranty

The warranty period is 6 months starting from first use, however not longer than up to the expiration date indicated on the packing as compression stockings are subject to a material and wearing caused aging. Made-to-measure garments are products for immediate use and are not suitable for storage. They are not provided with a best-before-date.

Please retain the fabric label and the packaging label.

The warranty/guarantee also extends to the use of COMPRESSANA accessories.

Risk category/medical devices

Medical compression stockings are medical devices in accordance to the Medical Devices Directive and correspond to the risk class 1. In the event of complaints and returns connected to the product, such as damages to the knitted fabric or a

faulty fit, please contact your specialised medical retailer directly. The manufacturer or the responsible authority of the member state is to be notified only in the event of a severe incident that leads to a significant worsening of health or death. Severe incidents are defined in article 2 no 65 of Regulation (EU) 2017/745 (MDR).

Quality mark RAL

The Medical Compression Hosiery Quality Assurance RAL-GZ 387/1 ensures safety by means of higher quality guidelines and test specifications. Products that meet the requirements are marked with this symbol on the insert label. Manufacturing in compliance to RAL is mandatory for prescription and reimbursement by statutory health insurance a.o. in Germany.

Wash and care references

Wash your compression stockings every day if possible. Washing the compression stockings serves not only the hygiene, but also restores form and stress and thus the full medical effect.

Wash before wearing. Please wash the stocking separately the first time (colour may bleed).

Turn inside out before washing.

Machine wash using a mild washing detergent and a gentle, delicate washing cycle at 40° C. We recommend the use of a special COMPRESSANA washing bag.

If you **wash by hand** it is advisable to use a mild washing detergent and lukewarm water. Rinse out well, do not wring.

May be dried in a tumble drier. Low temperature, maximum starting temperature is 60°C.

Do not dry clean.

Do not bleach.

Iron with a maximum iron sole temperature of 110 °C without steaming. Steam ironing may cause irreparable damage.

Do not use any softener.

Damaged footwear, inserts and horny skin at the feet can damage the stockings.

Stockings with a silicone band should be turned inside out before washing.

If Velcro fastener or zippers are fastened at the stockings close them carefully before washing.

Advice for fast drying: Wrap stockings in a towel and squeeze thoroughly e.g. stamp on the towel to squeeze the moisture out of the stockings and into the towel.

Indications

• Chronic venous diseases:

Improvement of venous symptoms, improvement of quality of life in chronic venous diseases, prevention and therapy of venous oedema, prevention and therapy of venous skin changes, eczema and pigmentation, dermatoliposclerosis and atrophie blanche, therapy for venous leg ulcer, therapy for mixed (arterial and venous) leg ulcers (taking into account contraindications), prevention of venous leg ulcer recurrence, pain reduction in venous leg ulcers, varicosis, initial phase after varicose therapy, functional venous insufficiency (in obesity, sedentary occupations, standing occupations), venous malformations

• Thromboembolic venous diseases:

Superficial venous thrombosis, deep vein thrombosis, condition after thrombosis, post-thrombotic syndrome, thrombosis prophylaxis in mobile patients

• Oedemas:

Lymphoedema, oedema during pregnancy, post-traumatic oedema, post-operative oedema, post-operative reperfusion oedema, cyclic idiopathic oedema, lipoedema, congestion due to immobility (arthrogenic congestion syndrome, paresis and partial paresis of the limb), occupational oedema (standing occupations, sedentary occupations), oedema due to medication if no change is possible.

• Other indications:

Obesity with functional venous insufficiency, inflammatory dermatoses of the legs, nausea, dizziness in pregnancy, congestive disorders in pregnancy.

In case of pregnancy, changing abdomen circumferences mean that a pregnancy body part should be exclusively used for treatment with pantyhose.

Following burns and for scar treatment, we recommend using our special scar therapy products.

The class of compression is determined by the attending doctor.

For hygienic reasons, this product is intended for one person only.

Contraindications

• Absolute contraindications:

Advanced peripheral arterial disease (if one of these parameters applies: ABPI < 0.5, ankle artery pressure < 60 mmHg, toe pressure < 30 mmHg or TcPO2 < 20 mmHg instep). Decompensated heart failure (NYHA III + IV), septic phlebitis, phlegmasia cerulea dolens, acute bacterial, viral or allergic inflammation, swelling of the extremities for unknown reasons.

• Relative contraindications:

Pronounced weeping dermatoses, intolerance to pressure or of any of the components of the product, severe sensory disorders of the extremities, advanced peripheral neuropathy (e.g. diabetes mellitus), primary chronic polyarthritis, peripheral arterial disease (PAD) stage I/II, malignant lymphoedema.

If you observe these contraindications, COMPRESSANA can not be made liable for the consequences.

Risks and side effects:

Wear compression stockings only after consulting your doctor and observe the instructions of the medical personnel. Medical compression stockings can cause skin necroses and stress damage on peripheral nerves, in particular with inappropriate handling. If pain or negative changes (skin provoking) during ordered application should arise, immediately contact your doctor.

In case of sensory and/or wound healing disturbances (e.g. diabetic diseases) implicating a skin lesion, badly healing impairments can develop unnoticed. Patients of this group require special control as well as a specific instruction about particular risks and precaution possibilities.

If there are incompatibilities materials and substance of content please consult your doctor, pharmacist or specialist retailer before use.

The manufacturer is not responsible for damage and injuries caused by inappropriate handling, non-observance of the instructions or misuse.

Disposal

There are no special disposal criteria. Compression products can be disposed with household refuse and the packaging material with paper recycling.

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© COMPRESSANA GmbH · Art.-Nr. 0838/04
COMPRESSANA GmbH