SN PPER

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Intended purpose:

To support the veins and lymph vessels, compression stockings fit tightly like a second skin. A donning aid can make the application much easier. Owing to a special construction and slidability putting on and taking off according to the SNAPPER CONCEPT is done quickly and without much strength required.

The very smooth material of COMPRESSION SNAPPER reduces frictional resistance and thus protects the skin from injury and irritation. The knitted stocking fabric is also protected from damage caused by mechanical impact.

In addition usage of special gloves with a grip on the palm is favourable, as recommended in the user manual of the hosiery manufacturers.

COMPRESSION SNAPPER makes the start into the compression therapy easier and ensures that the stockings are donned correctly. COMPRESSION SNAPPER is suitable for people who prefer an easier way of donning or people in special life circumstances, e.g. limited mobility.

Space-saving – ideal for travelling.

Detailed instruction!

Our partners in pharmacies or medical specialized trades will demonstrate how to use the COMPRESSION SNAPPER and will be on hand with help and advice for the first instruction. In the beginning practice donning and doffing with the COMPRESSION SNAPPER several times. The following illustrated instructions for donning will help you to perfect the handling. After a short time you will gain routine in the handling.

Special tips for handling of stockings with donning aids:

- Never pull the stockings at the top or at the top grip upwards.
- Upend the silicone beaded band before donning.
- In case of open wounds in the area of application, ensure that the wound is properly treated and covered.

For hygienic reasons, this product is intended for one person only.

Do never leave COMPRESSION SNAPPER on the floor. Considerable danger of slipping!

Donning:

1. Unfold COMPRESSION SNAPPER.
2. Fold the top flap inside in order to form a pocket.
3. Lay down the COMPRESSION SNAPPER on the floor and slip your foot into the pocket. To prevent shifting of the donning aid you can place the other foot on the bottom flap.
4. Next it is favourable to turn the stocking halfway inside out. Simply pull the stocking over the foot and over the heel.
5. Now use special gloves with grippy palms, if possible. Reach for the out sticking part of the COMPRESSION SNAPPER and pull it out. Simultaneous repeatedly bending of your toes up and down makes it easier.
6. After that raise the stocking upwards step by step, preferably with the help of stocking gloves. Evenly distribute the stocking on foot and leg. Watch out that that stocking is smoothed out and there are no creases.

Doffing:

Special gloves facilitate doffing of stockings. Push the stocking down the leg by sliding the stocking step by step downwards with the help of the special gloves.

Material composition:

100% Nylon / Polyamide

Wash references:

For simplest cleaning of COMPRESSION SNAPPER wipe with a damp rag.

Hand wash (approx. 30° C).

Use mild washing detergent (COMPRESSION CLEAN Art. No. 0121).

Do not tumble dry. Air-dry or wipe dry with a cloth.

Do not bleach.

Do not dry-clean.

Do not iron. Do not expose to heat (e.g. radiator).

Disinfection:

If used from different persons / users, e.g. in local surgeries, treatment facilities or nursing a wipe disinfection can be carried out with a surface disinfectant. Observe the instructions of the disinfectant.

Durability / Duration of use / Storage:

Store dry under hygienic conditions. Avoid heat, direct sunlight and moisture. The application time depends on the intensity of use and the type of fabric or thickness of the stockings. Regular use results in a material-related fair wear and tear.

Safety notice:

Please contact your specialist retailer or pharmacy in the event of damage. Do not repair the product yourself – this may affect its quality and effectiveness. Only use the donning aid when sitting. Carefully read the instructions for use and follow them. Keep the instructions in a safe place. The manufacturer does not accept any liability for damage / injuries resulting from improper use or misuse. Store COMPRESSANA SNAPPER away from children.

Indications:

- Paralysis
- Age-related reduction in strength
- Osteoarthritis / rheumatism
- Morbid obesity
- Extensive stiffening of the spine / hip / knee
- Degenerative disorders of the hands / in the area of the hand
- Consequence of injuries/amputations

Relative contraindications:

Absolute immobility.

Side effects:

There are no known side effects when this product is used correctly. However, if you notice any adverse changes during the prescribed use, please consult your doctor / pharmacy or medical retailer immediately. In the event of complaints related to this product, please contact the pharmacy / medical retailer directly. Only serious incidents that may lead to a major deterioration in the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR)

Disposal:

There are no specific criteria for disposal. All components when worn-out or at the end of service life or lack of the ability to function can be disposed of with household waste, the packaging material with waste paper recycling.

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COMPRESSANA GmbH