EASY

Ärmel Anziehhilfe
Sleeve fitting aid
Aide à l'enfilage
de la manche
Strumento di aiuto
per infilare maniche
Auxílio de colocação
de manga

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INTENDED PURPOSE:

COMPRESSANA EASY SLEEVE makes it easier to put on compression sleeves and elbow bandages. The slippery material allows you to pull on and fit high compression class sleeves easily and without frictional resistance.

The use of **COMPRESSANA EASY SLEEVE** in combination with stocking gloves is recommended. These not only make it easier to distribute and position the sleeve, but also protect the knitted fabric from damage (e. g. by sharp-edged fingernails or jewellery).

The fitting aid makes daily life with compression therapy easier and ensures that sleeves are correctly fitted. **COMPRESSANA EASY SLEEVE** sleeve fitting aid is suitable for anyone with special life circumstances, such as limited mobility, or simply to make fitting easier.

Space-saving – ideal for travelling.

Fitting aids can be prescribed if indicated.

Donning and doffing aids are recommended or prescribed by the doctor if the donning of the stockings without therapeutic appliances is too difficult. For example in cases of illnesses or disabilities with considerable dysfunctions of the of the mobility as well as for people with limited finger grip.

TEST IT FOR FREE!

If you are not sure if the **COMPRESSANA EASY SLEEVE** is a real alleviation you are welcome to test the donning and doffing aid without any engagement.

THOROUGH ADVICE AND INSTRUCTION!

The trained staff of our partners in pharmacy and specialist medical trades will be happy to advise you and demonstrate how to use the **COMPRESSANA EASY SLEEVE** and will be on hand with help and advice for the first instruction. In the beginning practise donning and doffing with the **COMPRESSANA EASY SLEEVE** several times. The following illustrated instructions for donning will help you to perfect the handling. After a short time you will gain routine in the handling.

The following illustrated instructions will help you master using the sleeve.

SPECIAL TIPS FOR HANDLING OF COMPRESSION PRODUCTS WITH FITTING AIDS:

- In case of open wounds in the area of application, ensure that the wound is properly treated and covered.

For hygienic reasons, this product is intended for one person only.

PUTTING ON:
Simple way to put on
compression sleeves.

Slip your arm into the
COMPRESSANA EASY SLEEVE
fitting aid.

Pull the compression sleeve
over the slippery material
of the COMPRESSANA EASY
SLEEVE fitting aid.

Pull the compression sleeve
over the fitting aid and onto
your arm. Make sure the sleeve
is positioned correctly, without
creases or overstretching.

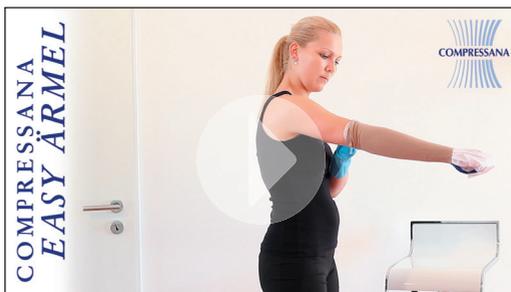
Use the loop to pull the
fitting aid out from under the
compression sleeve. To do so, put
the loop around a fixed or static
object (eg. a door handle).

REMOVAL:

GRIP stocking gloves make it easier to remove compression sleeves. Holding the woven fabric with the grips of the glove, slide the sleeve from your arm in stages.

**Do never leave COMPRESSANA EASY SLEEVE on the floor.
Considerable danger of slipping!**

**Watch COMPRESSANA EASY SLEEVE in action:
www.youtube.de/compressana
or use the QR-Code with your mobile**

MATERIAL COMPOSITION:

Polyamide / polyester mix

WASH REFERENCES AND TEXTILE CARE SYMBOLS:

For simplest cleaning of COMPRESSANA EASY SLEEVE wipe with a damp rag.

Hand wash (approx. 30° C).

Use mild washing detergent (COMPRESSANA CLEAN Art. No. 0121).

Do not tumble dry. Air-dry or wipe dry with a cloth.

Do not bleach.

Do not dry-clean.

Do not iron. Do not expose to heat (e.g. radiator).

DISINFECTION:

If used from different persons/users, e.g. in local surgeries, treatment facilities or nursing a wipe disinfection can be carried out with a surface disinfectant. Observe the instructions of the disinfectant.

DURABILITY / DURATION OF USE / STORAGE:

Store dry under hygienic conditions. Avoid heat, direct sunlight and moisture. The application time depends on the intensity of use and the type of fabric or thickness of the stockings. Regular use results in a material-related fair wear and tear.

Check COMPRESSANA EASY SLEEVE before use and application thoroughly for defects and proper functioning.

SAFETY NOTICE:

Please contact your specialist retailer or pharmacy in the event of damage. Do not repair the product yourself – this may affect its quality and effectiveness. Only use the donning aid when sitting. Carefully read the instructions for use and follow them. Keep the instructions in a safe place. The manufacturer does not accept any liability for damage/injuries resulting from improper use or misuse. Store COMPRESSANA EASY SLEEVE away from children.

INDICATIONS:

- Paralysis • Age-related reduction in strength • Osteoarthritis / rheumatism
- Morbid obesity • Extensive stiffening of the spine/hip/knee • Degenerative disorders of the hands/in the area of the hand • Consequence of injuries/amputations

RELATIVE CONTRAINDICATIONS:

Absolute immobility.

SIDE EFFECTS:

There are no known side effects when this product is used correctly. However, if you notice any adverse changes during the prescribed use, please consult your doctor / pharmacy or medical retailer immediately. The manufacturer does not accept any liability for damage /injuries resulting from improper use or misuse. In the event of complaints related to this product, please contact the pharmacy / medical retailer directly. Only serious incidents that may lead to a major deterioration in the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR)

DISPOSAL:

There are no specific criteria for disposal. All components when worn-out or at the end of service life or lack of the ability to function can be disposed of with household waste, the packaging material with waste paper recycling.

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COMPRESSANA GmbH