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COMPRESSANA ULCER SYSTEM

COMPRESSANA ULCER INTRA

Instructions for use

Mode d'emploi

Brugsanvisning

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Navodila za uporabo

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GB

Purpose

The COMPRESSANA ULCER SYSTEM is a medical compression stocking system for the treatment of ulcus cruris and consists of two compression stockings that can be combined:

ULCER INTRA*: The understocking with y-heel indication and closed foot tip serves to fixate the wound dressing or bandage. When pulling it on, ensure that the stocking has no wrinkles and that the bandage or wound dressing does not slip. The understocking can be worn on the affected leg day and night for continuous compression. Each package includes a second ULCER INTRA stocking for changing.

ULCER FORTE: The pull-over stocking with open toe serves to enhance the compression effect. The stocking is put on over the understocking and worn during the day.

Tips for use

Fingernails and jewellery (e.g. sharp-edged rings) can damage the fine knitted fabric. Therefore it is best to use COMPRESSANA GRIP stocking gloves to protect your stockings when putting them on. With foot and corneal care, correctly cut toenails and the right foot-wear, you can avoid damaging your stockings.

Putting on (image 1 - 7)

1. To put on the understocking with y-heel indication and closed toe, slip it over your forearm and reach into the foot area from the inside with your fingers. By pulling it out, turn the stocking inside-out up to the heel. Stretch the foot opening with both hands and slide the foot section over the foot up to the instep.
2. Gradually slip the understocking over the foot - simultaneously turning it back over to the right side. Make sure that the heel knit with the y mark is on the heel, as shown in image 12.
3. Now slide the understocking evenly and without wrinkles upwards, without twisting or pulling at the edges, and flatten it against the leg with your flat hand at the same time. If you have pulled the understocking too far upwards, simply push it back to the ankle joint and then pull it upwards with less force.
4. Now pull the included COMPRESSANA EASY gliding aid for open toe over the foot in such a way that the shorter flap lies over the instep/back of the foot and the longer flap lies against the sole.
5. Then pull the overstocking over the COMPRESSANA EASY gliding aid until the knitted stocking is in the heel position. The end of the donning aid in the toe area must protrude from the stocking opening at the front.
6. Now slide the entire overstocking upwards in sections towards the knee and simultaneously smoothen it against the leg with your flat hand. For optimal wearing comfort, position the end edge of the overstocking approx. 2 cm below the end edge of the understocking.
7. Finally, remove the EASY gliding aid by pulling it out of the front of the foot part.

Taking off (image 8 - 10)

Do not take off both stockings at the same time, but first fold down the overstocking with the open tip of the foot and strip it down the lower leg. Then carefully take it off, pulling it over the heel, instep and foot. Repeat these steps to take off the understocking.

Notes on ULCER INTRA understockings (image 11 - 12)

11. Please note that the toe seam of the understocking should be on the outside. This protects the sensitive skin in the toe area.
12. Place the understocking according to the knitted y-heel mark and ensure that it does not wrinkle.

Please note:

Only wear your compression stockings according to a doctor's prescription. The stockings should only be provided by suitably trained personnel after instruction. Changes to the leg caused by illness or therapy make a new measurement and, if necessary, new production necessary.

Your COMPRESSANA compression stockings are largely resistant to oils, ointments, skin moisture and other environmental influences. Certain combinations may affect the durability of the knitted fabric. Therefore, a regular check in a pharmacy or in a medical specialist shop is recommended.

In the event of damage, please contact your pharmacy or specialist dealer. DO NOT repair the stockings yourself - this can impact the quality and medical effect. For hygienic reasons, this product is intended for one person only.

*ULCER INTRA understockings with closed toe and y-heel indication are also available separately (additional delivery set).

Material composition

For precise information please see the label sewn in your compression stockings. All yarns used are resistant to sunlight and natural rubber. Compressana compression stockings are free of latex and rubber.

Durability and storage options

Store at room temperature and in a dry place. Avoid heat and moisture. In the case of series productions, the maximum storage duration is 36 months. A 6-month service life of the compression garments follows that period. The time of use of the medical product is printed onto the box label with an hour glass symbol.

Tailor-made items are products that are designed for immediate use. Since the body size and dimensions may change due to the progression of an illness or specific personal circumstances, tailor-made products are designed for 6 months of use. After 6 months of use, a renewed measuring of the

body dimensions by a medical specialist becomes necessary. In case of item damage please contact your specialist retailer or dispensing chemist. Do not attempt to mend stockings yourself as this could affect the medical effect.

Warranty

The warranty period is 6 months starting from first use, however not longer than up to the expiration date indicated on the packing as compression stockings are subject to a material and wearing caused aging. Made-to-measure garments are products for immediate use and are not suitable for storage. They are not provided with a best-before-date.

Please retain the fabric label and the packaging label.

Risk class/medical product

Medical compression stockings are medical products and correspond to risk class 1.

In the event of product-related claims, such as damage to the knitted fabric or defects in the fit, please contact your medical dealer/healthcare professional directly. Only serious incidents that could lead to a significant deterioration in health or death must be reported to the manufacturer and the competent authority of the Member State. Serious incidents are defined in Article 2 (65) of Regulation (EU) 2017/745 (MDR).

Quality Mark RAL

The Medical Compression Hosiery Quality Assurance RAL-GZ 387/1 ensures safety by means of higher quality guidelines and test specifications. Manufacturing in compliance to RAL is mandatory for prescription and reimbursement by statutory health insurance in Germany.

Washing and care instructions

Please note the care instructions on the textile label sewn into your compression garment or the printed information. Do up any fasteners (if there are any) before washing! We recommend turning compression garments inside out before washing. Wash compression garments separately when washing for the first time (they may lose colour). If possible, your compression garments should be washed every day on a gentle or delicate wash cycle (ULCER FORTE 40 °C, ULCER INTRA 60 °C). Tip: a laundry net provides additional protection for the knitted fabric. Please do not use fabric softener! When hand washing, rinse well and do not wring. We recommend the gentle COMPRESSANA special detergent. You can reduce drying time by placing the compression garment on a thick towel after washing, tightly rolling the towel up and firmly pressing out excess moisture. Then hang up the compression garment loosely. Do not leave the product in the towel

and do not dry it on a radiator or in the sun. Please use the delicates setting when drying in a tumble dryer (suitable for tumble-drying: see textile label or printed information on the fabric). COMPRESSANA compression garments are not suitable for dry cleaning.

Indications

Treatment of venous leg ulcers, treatment of mixed (arterial and venous) leg ulcers (taking into account contraindications), use instead of or as a supplement to compression bandages following oedema reduction, prevention of venous leg ulcer recurrence, pain reduction in cases of venous leg ulcers.

Contraindication

Advanced peripheral arterial disease (if one of these parameters applies: ABPI < 0.5, ankle artery pressure < 60 mmHg, toe pressure < 30 mmHg or TcPO₂ < 20 mmHg instep).

Decompensated heart failure (NYHA III + IV), septic phlebitis, phlegmasia cerulea dolens. In the following cases, the decision about treatment should be made considering the benefits and risks as well as choosing the most suitable compression option:

Pronounced weeping dermatoses, intolerance to the compression material, severe sensory disorders of the extremities, advanced peripheral neuropathy (e.g. diabetes mellitus), primary chronic polyarthritis.

If you do not observe these contraindications, Compressana assumes no liability.

In the case of open wounds in the treatment area, the wound area must first be covered with an appropriate wound dressing or bandage before the stocking system is donned. The skin in the treatment area must be checked regularly.

Risks and side effects

When used properly, no side effects are known. If, however, negative changes should arise during the prescribed application, please contact your doctor or medical specialist immediately. If an intolerance to the ingredients of this product is known, please consult your doctor before use.

The manufacturer is not liable for any damage or injury resulting from improper use or failure to follow the instructions.

Disposal

There are no special disposal criteria. Compression products can be disposed with domestic refuse and the packaging material with paper recycling.

Aids for putting on and off

COMPRESSANA put-on aids make it easier to put on and take off compression stockings. These are gentle on the skin and knit of the stockings. COMPRESSANA put-on aids are prescription aids.

COMPRESSANA EASY donning aid leg

The simple and inexpensive put-on aid made of gliding cloth makes it possible to put on stockings without frictional resistance. EASY also makes it easier to dress wound covers or bandages on the foot or leg

COMPRESSANA SNAPPER donning aid

The simple and inexpensive put-on aid made of gliding cloth makes it possible to put on stockings without frictional resistance. EASY also makes it easier to dress wound covers or bandages on the foot or leg

COMPRESSANA MASTER

the station for putting on and taking off

In the case of reduced mobility, grasping ability and mobility, the special metal construction of COMPRESSANA MASTER allows effortless dressing and undressing in one - simply while sitting, without strenuously bending down and without any effort of the hands.

COMPRESSANA GRIP stocking gloves

With COMPRESSANA GRIP stocking gloves you have a better grip to spread the knitted fabric evenly on the leg. It is also easy to correct the position of stretched or slipped stockings.

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© COMPRESSANA GmbH · Art.-Nr. C201/01
COMPRESSANA GmbH