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## Knee-high support socks with cotton

supports and relieves your legs, prevents vein problems and varicose veins, helps against heavy, tired and aching legs

### COMPRESSANA TWIST

Strong support appr. 18 mmHg, 58% Cotton, 26% Polyamide, 16% Elastane  
• fine, elegant knit • gentle on skin, breathable

### COMPRESSANA TWIN

Extra strong support appr. 21 mmHg, 65% Cotton, 22% Polyamide, 13% Elastane

• strong and hard-wearing • with soft, yarn-dyed cotton

Available with open or closed toe. If you have difficulties with putting on the stockings, a feeling of pressure or tightness, or foot diseases, choose the open toe.

### Knee-high support socks with silver fibre

#### COMPRESSANA X-STATIC

Strong support appr. 18 mmHg, 54% Polyester, 34% Polyamide, 6% Elastane,  
6% X-Static / Silver

Properties of the silver fibre: The permanent fixation of silver on the surface of a high-performance fibre has the following properties:

- antimicrobial • kind to the skin • adapts to climatic conditions • conductive / anti-static

The decreasing pressure gradient towards the heart enhances the properties of the silver fibre: • prevents varicosis and swelling • helps against heavy, tired and aching legs • activates blood circulation

### Knee-high support socks with merinowool

#### COMPRESSANA THERMO lite

Strong support appr. 16 mmHg, 46% Wool, 42% Polyamide, 8% Elastane  
• with soft merinowool for cold days • supports and relieves your legs • helps against heavy, tired and aching legs

### Socks with support and brace effect

#### COMPRESSANA aktiv

Strong support app. 18 mmHg, 58% Cotton, 26% Polyamide, 16% Elastane

- support and brace effect for foot and ankle joint • offers better stability • can prevent swellings and relieve pain • reduces the risk of injury • suitable for work, sports and leisure activities

Due to the confined spaces in aeroplanes, buses, trains or cars your legs often stay motionless and in angled position for hours. The blood is pooling in the legs and the venous blood return is hampered. COMPRESSANA below-knee stockings ideally support the veins in your legs and maintain the blood circulation, especially while travelling.

### Application / Donning and maintenance instructions

Please observe the information on the packaging, the enclosed product leaflet, or the instructions of your physician, pharmacist or medical specialist. In the event of difficulties in donning or using the product, you can find more in-depth instructions/information and accessory-related videos at [www.compressana.de/downloads](http://www.compressana.de/downloads) and youtube/compressana. Alternatively, please consult your medical specialist / pharmacist / specialist retailer. COMPRESSANA donning aids and special-purpose gloves can simplify donning and removing the stockings.

### Durability / Application period / Storage

Store in a dry, dark place at room temperature to protect against UV radiation. Avoid direct sunlight, heat and moisture. The application period depends on the intensity of usage. Depending on the application and material, the stockings will suffer wear, tear and natural deterioration during regular use. Check the stockings thoroughly for flaws and fitness for purpose before use. In the event of any damage, consult your specialist retailer or pharmacist..

### Risks / Side effects

When used incorrectly or worn in the wrong size or fit, support stockings can cause skin necrosis, oedema or pressure damage to peripheral nerves. If you notice any pain or negative changes (e.g. skin irritation, constrictions) during use, remove the stockings and **consult your doctor or pharmacist / medical specialist without delay**. In the event of somatosensory disorders and/or wound-healing disorders (e.g. diabetic illnesses) in combination with skin lesions, poorly-healing wounds can occur unnoticed. People in this category require special monitoring and specific instructions regarding specific risks and preventive methods.

Do not wear the stockings in the event of open wounds or suppurating dermatosis/skin diseases. Alternatively, ensure that any wounds are properly dressed before donning the stockings.

The manufacturer accepts no liability for any damage or injury due to improper use, non-adherence to the instructions or as the result of using accessory products not manufactured by COMPRESSANA. Inform your doctor if the effect is insufficient to ease any existing complaints.

**For hygienic reasons, this product is intended for one person only.**

### Disposal

There are no special criteria for disposal. The stockings can be disposed of via the household waste, the packaging via the paper recycling.

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