

Compression Wrap

Adjustable compression system

Based on their extremely short-stretch properties, Compression Wrap products exert very high working pressure and low resting pressure. The double-sided touch fasteners and helpful gripping zones ensure easy donning and adjustment in the event of fluctuations in body circumference. Special indentations on the tabs ensure complete closure and prevent the formation of “window oedemas.” Special feature: the product is reversible, with black on one side and beige on the other, to suit the individual patient’s preference.

Features

- + variable compression 30 – 60 mmHg
- + can be individually adjusted and cut to size when treating fluctuating leg circumferences
- + special indentations ensure the perfect overlap of the tabs, thus preventing the formation of “window oedemas”
- + gripping zones at the upper and lower end make donning easier
- + the product is reversible, with black on one side and beige on the other, to suit the individual patient’s preference
- + **Compression Wrap segments can also be combined with Soft-Compress products**

Material composition

Outer material: 88 % polyamide, 12 % elastane
Inner material: 100 % neoprene

Contraindications

See page 37

Colours



Beige / Black



very high working pressure



reversible



standard sizes



Special indentations

Thanks to this fixture on the tabs, the formation of “window oedemas” is prevented. The indentations ensure that the individual tabs overlap without gaps.



Helpful gripping zones

Practical zones at the upper and lower end make closing the tabs easier.



Individual colour selection

Thanks to the special material processing, the product is reversible, with black on one side and beige on the other.

Lower extremity

Compression Wrap variable compression system for the lower extremities are compression segments with short-stretch characteristics, intended to reduce oedema in the foot, lower leg or the entire leg. Equipped with adjustable straps, they can be adapted to fit various circumferences, even if those change during treatment. These are highly suited to patients' selfbandaging. Segments for the lower leg and foot can be combined.

NEW If required, wraps for the knee or the thigh can be added to treat the entire leg. Liner, available separately, is recommended as liner stockings.



Compression Wrap Thigh

- + special indentations allow for the perfect overlap
- + gripping zones for easier donning
- + can be worn both left and right
- + the product is reversible, with black on one side and beige on the other

NEW



Compression Wrap Knee

- + special indentations allow for the perfect overlap
- + gripping zones for easier donning
- + can be worn both left and right
- + the product is reversible, with black on one side and beige on the other

NEW



Compression Wrap Calf

- + with incorporated lining (fixing aid)
- + includes liner stocking
- + special indentations allow for the perfect overlap
- + gripping zones for easier donning
- + can be worn both left and right
- + the product is reversible, with black on one side and beige on the other



Compression Wrap Foot






- + special indentations allow for the perfect overlap
- + gripping zones for easier donning
- + can be worn both left and right
- + the product is reversible, with black on one side and beige on the other



Product range

Compression Wrap for lower extremities

Adjustable compression system

	Art. Length	Model	Unit	Standard sizes
	6000LF	Foot wrap	Each	
	Normal	NEW: Size 5		2 – 5
	Long NEW			
	6000LC	Calf wrap	Each	
	Normal			2 – 7
	Long			
	6000LCM	Calf wrap max.	Each	
	Normal			2 – 7
	Long			
	6000LK	Knee wrap	Each	
				2 – 7
	6000LT	Thigh wrap	Each	
	Normal			2 – 6
	6000LTg			
	Long			

Sizing charts

Compression Wrap for lower extremities

Adjustable compression system

Juzo Compression Wrap Foot

Art. 6000 LF/LFL		Standard sizes - Circumferences in cm					
		2 / S	3 / M	4 / L	5 / XL		
	cB	15.0 – 23.0	20.0 – 28.0	25.0 – 33.0	30.0 – 40.0		
	cA*	19.0 – 24.0	24.0 – 29.0	29.0 – 34.0	34.0 – 40.0		
	PA	normal	16	18	19.5	19.5	
		long	19	21	22.5	23	

* 6 cm proximal to the little toe metatarsophalangeal joint

Juzo Compression Wrap Calf

Art. 6000 LC/LLC/LCM/LLCM		Standard sizes - Circumferences in cm					
Lengths in cm		2 / S	3 / M	4 / L	5 / XL	6 / XXL	7 / XXXL
	PD AD	29.0 – 35.0	34.0 – 40.0	37.0 – 44.0	41.0 – 48.0	46.0 – 53.0	50.0 – 60.0
	normal < 40,0	35.0 – 41.0	40.0 – 46.0	44.0 – 51.0	48.0 – 55.0	53.0 – 60.0	60.0 – 70.0
	long > 40,0	18.0 – 21.0	21.0 – 24.0	24.0 – 27.0	27.0 – 31.0	31.0 – 35.0	36.0 – 47.0

Juzo Compression Wrap Knee

Art. 6000 LK		Standard sizes - Circumferences in cm					
		2 / S	3 / M	4 / L	5 / XL	6 / XXL	7 / XXXL
		30.0 – 36.0	36.0 – 42.0	42.0 – 48.0	48.0 – 55.0	55.0 – 62.0	62.0 – 70.0

Juzo Compression Wrap Thigh

Art. 6000 LT/LTg		Standard sizes - Circumferences in cm				
Lengths in cm		2 / S	3 / M	4 / L	5 / XL	6 / XXL
	PFG	43.0 – 70.0	54.0 – 81.0	65.0 – 92.0	75.0 – 102.0	85.0 – 112.0
	normal < 25,0	35.0 – 55.0	48.0 – 70.0	60.0 – 82.0	70.0 – 92.0	80.0 – 102.0
	long > 30,0					



Application videos:
Compression Wrap, lower leg




Compression Wrap, leg



Upper extremity

Compression Wrap variable compression system for the upper extremities are compression segments with short-stretch characteristics, intended to reduce oedema in the arm and hand. Equipped with adjustable straps, they can be adapted to fit various circumferences, even if those change during treatment. These are highly suited to patients' self-bandaging. Segments for the arm and hand can be combined. The hand wrap is particularly suited for treating oedema in the area of the back of the hand.



Compression Wrap Arm

- + with incorporated lining (fixing aid)
- + special indentations allow for the perfect overlap
- + can be worn both left and right
- + both sides of the product can be used; the colour can therefore be chosen individually

NEW

Compression Wrap Hand

- + special indentations allow for the perfect overlap
- + available with extender in left and right versions
- + the side can be chosen based on the preferred colour*

NEW

*Please note:

in order to achieve perfect wearing comfort, the side of the hand wrap (right or left) must be selected according to the colour. If the hand wrap is to be worn with beige on the outside, the desired colour must be chosen during the order. If black is preferred on the outside, the opposite side must be ordered.

Example:

Required: hand wrap, size M, for the left hand:

- Order in **beige**: 6000 UHL, size M, **left**
- Order in **black**: 6000 UHR, size M, **right**





Application video:
Compression Wrap, arm



Product range

Compression Wrap for upper extremities

Adjustable compression system

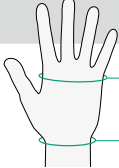
	Art.	Model	Unit	Standard sizes
	Length			
	6000UHL	Hand wrap left	Each	
	Beige: left Black: right			1 – 5
	6000UHR	Hand wrap right	Each	
	Beige: right Black: left			1 – 5
	6000UA	Arm wrap	Each	
	Normal			1 – 5
	Long			

Sizing charts

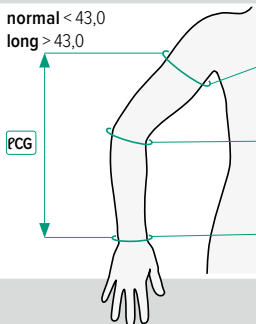
Compression Wrap for upper extremities

Adjustable compression system

Juzo Compression Wrap Hand

Art. 6000 UHR/UHL		Standard sizes · Circumferences in cm				
		1 / XS	2 / S	3 / M	4 / L	5 / XL
	cA	15.5 – 17.5	17.5 – 19.5	19.5 – 22.0	22.0 – 24.5	24.5 – 28.0
	cC	13.0 – 17.0	15.0 – 19.0	17.0 – 21.5	20.0 – 24.0	21.0 – 25.0

Juzo Compression Wrap Arm

Art. 6000 UA/UAG	Lengths in cm		Standard sizes · Circumferences in cm				
			1 / XS	2 / S	3 / M	4 / L	5 / XL
	normal < 43,0	cG	27.0 – 36.0	30.0 – 39.0	33.0 – 44.0	35.0 – 49.0	40.0 – 56.0
	long > 43,0	cE	23.0 – 32.0	24.0 – 34.0	26.0 – 36.0	29.0 – 42.0	31.0 – 46.0
		cC	16.0 – 19.0	17.0 – 20.0	18.0 – 22.0	20.0 – 26.0	21.0 – 29.0

Additional products

Compression Wrap

Adjustable compression system

Complementary additional products are available for Compression Wrap. They support patients in adapting their garments to individual requirements and are therefore perfectly suited for self-bandaging.



Compression Wrap
Touch Fastener

Art. 6000F2

Double-sided Compression Wrap touch fasteners make it easy to put on the products and to adjust them in the event of fluctuations in body circumference.




Compression Wrap
Extender

Art. 6000EXT

The Compression Wrap extender is used for the individual extension of a segment or pressure increase. It can be cut to size as required.

Product range

	Art.	Model	Unit
	6000F1	Touch fasteners	Set: 4 pieces
	Size: 40 mm	Foot wrap, hand wrap	•
	6000F2	Touch fasteners	Set: 4 pieces
	Size: 75 mm	Calf wrap	•
	6000F3	Touch fasteners	Set: 5 pieces
	Size: 85 mm	Calf wrap long, arm wrap	•
	6000F4	Touch fasteners	Set: 5 pieces
	Size: 95 mm	Knee wrap, thigh wrap, arm wrap long	•
	6000F5	Touch fasteners	Set: 3 pieces
	Size: 105 mm	Thigh wrap long	•
	6000F6	Touch fasteners	Set: 4 pieces
	Size: 50 mm	Foot wrap long	•
	6000EXT	Extender	Piece
	Size: 120 x 8.5 cm		•

Liner

Liner stockings for adjustable compression systems



Liner

Art. 6082

Liner stockings are liner stockings for use under any wrap system. They are gentle on sensitive skin and protect any dressing there may be while the compression system is put on and taken off. These black stockings come as a pair. Liner is available in two sizes each as calf-high or thigh-high stockings.

Colours



Black

Product range

	Art. 6082 Type	Model	Standard sizes
	AD	Below-knee stockings	
		Set with 2 stockings	1 – 2
	AG	Thigh-high stockings	
		Set with 2 stockings	1 – 2

Sizing chart

Juzo Liner

Art. 6082		Standard sizes · Circumferences in cm	
Lengths in cm		I / XS	II / S
	PG AG normal 80.0	50.0 – 75.0	76.0 – 99.0
	PD AD normal 45.0	29.0 – 48.0	48.0 – 69.0

INTRA

Liner stocking with SkinProtect® Silver

Adjustable compression system



INTRA Base layer below-knee stocking
Art. 0096, Art. 0086 (extraweit)

INTRA Base layer sock
Art. 0094

These soft-fabric stockings are especially comfortable to wear and are also suitable as a compression aid for patients with extremely sensitive skin. The stockings offer permanent skin protection during donning and doffing and, thanks to the knitted-in silver thread, also have an antibacterial effect. They are ideally suited for incorporating wound dressings or securing special pressure pads.

Colour



Silver



Product range

	Style	INTRA	Standard sizes
	AD	Base layer below-knee stocking SkinProtect® Silver	
	Art. 0096 Art. 0086	One-Size extra wide for calf circumference > 50 cm	• •
	AB	Base layer sock SkinProtect® Silver	
	Art. 0094	One-Size	•

Sizing chart

Art. 0096, Art. 0086 (extra wide)

series sizes · circumference in cm

INTRA Base layer below-knee stocking silver

< 50,0

extra wide > 50,0

Additional information

Adjustable compression system

When do we recommend which product?

ACS Light

- compression up to 40 mmHg
- with secondary conditions that require low pressure (e.g. mild PAD, peripheral neuropathy)
- for patients who cannot tolerate high compression
- for anatomical features that require increased material elasticity (e.g. pronounced skin flaps in the area of the thigh)
- for a temporary increase in compression for flat knit products

Compression Wrap

- compression 30 – 60 mmHg
- if very high working pressure is required (e.g. in cases of venous leg ulcers)
- for pronounced fibrosis
- for very pronounced lymphoedema (e.g. primary lymphoedema, lymphoedema in stages II - III)



Contraindications

SoftCompress, ACS Light and Compression Wrap

Absolute contraindications

- advanced peripheral arterial occlusive disease (if one of these parameters applies ABPI < 0.5, ankle artery pressure < 60 mmHg, toe pressure < 30 mmHg or TcPO2 < 20 mmHg instep). When inelastic materials are used, compression garments can still be worn with an ankle artery pressure between 50 and 60 mmHg with continual clinical supervision.
- decompensated cardiac insufficiency (NYHA III + IV)
- septic phlebitis
- phlegmasia coerulea dolens
- acute bacterial, viral or allergic inflammation
- swelling of the feet or legs with unknown cause

Relative contraindications

- Pronounced weeping dermatoses
- Incompatibility with pressure or product ingredients
- Severe sensitivity disorders in the extremities
- Advanced peripheral neuropathy (e.g. in cases of diabetes mellitus)
- Primary chronic polyarthritis
- Peripheral artery disease (PAD) stage I/II
- Malignant lymphoedema

Important:

If there are open wounds in the area to be treated, the affected area has to be covered with appropriate dressing or bandaging after consulting with a doctor, before SoftCompress or ACS Light can be applied.

Complementary products for oedema therapy

Skin care



Derma Lotion

Art. 0950 (500 ml)

Art. 0925 (125 ml)

Moisturising lotion with 4 % urea for targeted skincare in cases of lymphoedema and lipoedema



Skin Protect

Art. 0757

Skin regeneration and acute care of stressed, sensitive areas of the skin, even in cases of venous disorders. The antimicrobial effect of silver reduces the risk of infection.



Anti-itch spray

Art. 0583 (150 ml)

The acute care against itching. The active ingredient polidocanol soothes the skin and can prevent itching for several hours.

Stocking gloves



Stocking gloves Super-Grip

Art. 0112

Extra fine textile glove with latex finish for gentle donning and better spreading of the knitted fabric

Size II (S)* = up to 8 cm handwidth

Size III (M)* = up to 9 cm handwidth

Size IV (L)* = up to 10 cm handwidth

Size V (XL)* = up to 11 cm handwidth



Stocking gloves Top-Grip

Art. 0108

Breathable, sensitive and durable textile gloves made of nylon with nitrile coating

Size II (S)* = up to 8 cm handwidth

Size III (M)* = up to 9 cm handwidth

Size IV (L)* = up to 10 cm handwidth

Size V (XL)* = up to 11 cm handwidth

* The size is determined by measuring the hand width at the finger baseline

Laundrying care



CLEAN detergent

Art. 0121 (250 ml)

Particularly mild and economically concentrated detergent for gentle cleaning of compression garments.



Laundry bag

Art. 1007

Protects your stockings and tights during machine wash quick and easy to fill by means of zipper.



Silver laundry bag

Art. 0891

With pure silver fabric (0% nano particles) to combat unpleasant smelling laundry, even at low temperatures.

Skin glue



Adhesive lotion

Art. 0126 (Roll-On 60 ml)

The skin-friendly, residue-free removable adhesive lotion gives stockings, supports and other textiles safe and permanent hold. After wearing the adhesive lotion is removable with water or with a moist cloth within seconds.